VERMEIREN

Eris

INSTRUCTION MANUAL
MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ANUALEDIISTRUZIONIMAN
UALDEINSTRUCCIONESINS
TRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

EN

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Manual de instrucciones

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Contents

1	Product description	3
1.1	<i>Intended Use</i>	3
1.2	<i>Safety instructions</i>	4
1.3	<i>Technical specifications</i>	5
1.4	<i>Accessories</i>	6
1.5	<i>Components</i>	7
1.6	<i>Location identification plate</i>	7
1.7	<i>Explanation of symbols</i>	8
1.8	<i>Expected lifespan</i>	8
2	Use	9
2.1	<i>Remarks on electromagnetic compatibility (EMC)</i>	9
2.2	<i>Carrying the scooter</i>	10
2.3	<i>Assembly and disassembly of the scooter</i>	10
2.4	<i>Operating the brakes</i>	11
2.5	<i>Transfer in and out the scooter</i>	11
2.6	<i>Correct position in the scooter</i>	11
2.7	<i>Driving the scooter</i>	12
2.8	<i>Driving the scooter on ramps</i>	14
2.9	<i>Driving the scooter over obstacles</i>	15
2.10	<i>Operator control</i>	15
2.11	<i>Neutral</i>	16
2.12	<i>Transport</i>	16
2.13	<i>Automatic power shut down</i>	17
2.14	<i>Charging the batteries</i>	17
3	Installation and adjustment.....	19
3.1	<i>Tools</i>	19
3.2	<i>Manner of delivery</i>	19
3.3	<i>Adjusting the steering unit</i>	19
3.4	<i>Adjusting the seat</i>	20
3.5	<i>Folding the backrest</i>	21
3.6	<i>Adjusting the rear view mirrors</i>	22
3.7	<i>Adjusting the armrests</i>	22
3.8	<i>Tyre changing</i>	22
3.9	<i>Thermal fuses</i>	23
3.10	<i>Anti-tipping</i>	24
3.11	<i>Changing the battery</i>	24
4	Maintenance	25
4.1	<i>Instructions for reuse</i>	25
5	End of use	26
6	Declaration of conformity	26

Preface

First of all we want to thank you for putting your trust in us by selecting one of our scooters.

The expected lifetime of your scooter is strongly influenced by the care and maintenance of the scooter.

This manual will help you get acquainted with the operation of your scooter.

Following the user instructions and the maintenance instructions are an essential part of the warranty.

This manual reflects the latest product developments. Vermeiren has the right to introduce changes without the obligation to adapt or replace previously delivered models.

For any further questions, please consult your specialist dealer.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of following information. Please consult this website regularly since the printed version may be out of date.

- This manual
Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

- EC declaration of conformity
For user and specialist dealer

- Instructions for mounting and demounting of optional parts
For specialist dealer

- Service manual
For specialist dealer

Contact details

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility.

1 Product description

1.1 Intended Use

The scooter is intended to comfortably transport people. This scooter is designed to transport 1 person. Do not carry passengers on your scooter (including children).

The scooter is designed to transport adult people only, no goods, nor any persons under the age of 16.

Neither should it be used by people obviously suffering from physical or mental limitations that render them incapable of safely handling the scooter in traffic.

These limitations could involve:

- hemiplegics or paraplegics
- mental disorders
- visual impairment

The scooter is classified as class B.

The scooter is suited for outdoor use by the occupant.

When providing for individual requirements:

- body size and weight (max. 140 kg)
- physical and psychological condition
- residential circumstances
- environment

should be taken into consideration.

Your scooter should be used on flat surfaces where all four wheels are touching the ground and where there is sufficient contact to propel the wheels equally.

You should practice for use on uneven surfaces (cobblestones, etc.), slopes, curves and to get past obstacles (curbs, etc.).

The scooter should not be used as a ladder, nor is it a transport for heavy or hot objects.

When using your scooter on streets or footpaths, local laws and regulations apply.

The scooter can be used on sidewalks, urban road. Under no circumstances the scooter may be used for driving on larger roads or expressways.

Use only Vermeiren approved accessories.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

Compliance with the user and maintenance instructions are an essential part of the warranty conditions so we recommend that you read the following pages very carefully.

Visually impaired people can contact the dealer for the instructions for use.

Note: This product is suitable and approved for reuse by another patient, when delivered via a rental service. Follow the instructions in §4.1

1.2 Safety instructions

-  Use only Vermeiren approved accessories.
-  Before getting on or off, taking apart or transporting your scooter, turn the start key to the “off” position.
-  When transporting the scooter, no persons may be transferred along with it.
-  Test the effects on the handling of the scooter when its center of gravity is shifted, for example on up or down gradients, on laterally sloping ground or when driving over uneven terrain.
-  Do not drive on roads with heavy traffic or roads that are muddy, gravelly, bumpy, narrow, snowed over, icy or canal towpaths not guarded by any fence or hedge.
-  Keep away from places where you might get the wheels stuck.
-  Pay attention that your clothes do not tangle in the wheels.
-  When picking up objects lying in front of, next to, or behind the scooter, take care that you do not lean too far sideways: risk of tipping over.
-  Avoid putting your scooter into the free-wheeling position on slopes.
-  Never reverse uphill.
-  Reduce speed when you go round corners.
-  Do not take the scooter onto escalators.
-  While driving, hold the handgrips with both hands.
-  Put your legs and feet on the surfaces provided for them during the drive.
-  Do not use your scooter in snowing, misty or heavy windy conditions.
-  When storing or parking your scooter outside, protect it with a shrouding cover against humidity.
-  High levels of humidity or very cold conditions can reduce the performance of your scooter.
-  Use your scooter strictly in accordance with regulations. Avoid driving straight over obstacles (e.g. step, edge of the curb) or down from high ledges.
-  Obey traffic regulations when driving on public roads. Please take other road-users into consideration.
-  As is the case with any other vehicle, you must never be under the influence of alcohol while driving your scooter. This also applies to indoor driving.
-  When traveling outdoors, adapt your driving to weather and traffic conditions.
-  When driving in the dark, wear bright clothing or clothing with reflectors to be more visible, and make sure that the reflectors on the scooter are clearly visible and drive with lighting on.
-  Make sure that the lights of your scooter are free from dirt and/or other objects that could hide them.
-  Never use your scooter as a seat in an automobile or other vehicle.
-  Check that the profile depth of the tyres is adequate.
-  If your scooter has pneumatic tyres, be careful to inflate them to the correct pressure (see *pressure indication on the tyres*) for optimal steering and stability of the scooter.
-  Make sure that the maximum load of the scooter is not exceeded.
-  Do not overload the shopping basket. The maximum load of the shopping basket is 5 kg.
-  We advise not to use power strip and extension cord. Use the battery charger directly on the supply socket. We advise not to use supply sockets with a time switch.

1.3 Technical specifications

Technical terms below are valid for the scooter in standard settings and optimum environmental conditions. If other accessories are used, the tabulated values will change. Changes in outdoor temperatures, humidity, uphill, downhill, soils and battery levels can reduce output.

Brand	Vermeiren		
Address	Vermeirenplein 1/15, B-2920 Kalmthout		
Type	Rear wheel drive scooters, Class B		
Maximum occupant mass	140 kg		
Model	Eris		
Description	Dimensions		
Max. speed	6 km/h	10 km/h	
Continuous driving distance range*	35 km (Battery 38 Ah) 45 km (Battery 50 Ah)	29 km (Battery 38 Ah) 37 km (Battery 50 Ah)	
Length	1185 mm		
Width	570 mm		
Height	1150 mm		
Seat width	435 mm		
Folded / dismantled length	950 mm		
Folded / dismantled width	570 mm		
Folded / dismantled height	600 mm (seat removed; steering unit folded)		
Total mass	75 kg		
Mass of heaviest part (that can be dismantled or removed)	57 kg (without seat) or 46 kg (without seat and batteries)		
Masses of parts that can be dismantled or removed	Seat: 11 kg; Batteries: 26,5 kg		
Static stability downhill	9°		
Static stability uphill	9°		
Static stability sideways	9°		
Dynamic stability	6°		
Maximum safe slope	6°		
Ground clearance	50 mm (anti-tipping)		
Obstacle climbing	75 mm		
Seat plane angle	6°		
Effective seat depth	360 mm		
Seat surface height at front edge (measured from ground)	595 mm - 645 mm		
Seat surface height at front edge (measured from footplate)	450 mm - 500 mm		
Height footplate	145 mm		
Backrest angle	25°		
Backrest height	340 mm		
Distance between armpad and seat	180 mm		
Front location of armrest structure	270 mm		
Motor	Nom. 320 Watt, electromagnetic brakes		
Batteries	38 Ah	50 Ah	
Controller	70 A		

Brand	Vermeiren		
Address	Vermeirenplein 1/15, B-2920 Kalmthout		
Type	Rear wheel drive scooters, Class B		
Maximum occupant mass	140 kg		
Model	Eris		
Description	Dimensions		
Nominal voltage (battery)	2 x 12V ---		
Degree of protection	IPX4		
Battery charger	6 Amp (external)	8 Amp (external)	
USB connection	No		
Battery charger protection class	IPx1		
Battery charger insulation class	II		
Minimum turning diameter	2900 mm		
Reversing width	2900 mm		
Diameter rear wheels (number)	3.00-4 air (2)		
Tyre pressure, rear (driving) wheels	Max. 3,5 bar		
Diameter steering wheels (number)	3.00-4 air (2)		
Tyre pressure, steering wheels	Max. 3,5 bar		
Handle bar	Speed control lever		
Lights	Serial		
Indicator light	Serial		
Mirrors	Optional		
Storage and use temperature	+5 °C to +40 °C		
Operating temperature of the electronics	-10°C to +40°C		
Storage and use humidity	30%		
Tested and meets requirements of	ISO 7176-14:2008		
Resistance to ignition acc. to	EN1021-2		
We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / 1,5°.			
* The theoretical driving distance will be reduced if the scooter is used frequently on slopes, rough ground or to climb curbs. Maximum driving distance is tested according ISO 7176-4 under ideal circumstances.			

Table 1: Technical specifications


1.4 Accessories

The following accessories are available for the scooters Eris:

⚠ WARNING: Risk of injury - Be sure that crutches are attached very well and cannot fall on the user.


- Crutch holder
- Scooterpac
- Topcase
- Rollator holder

1.5 Components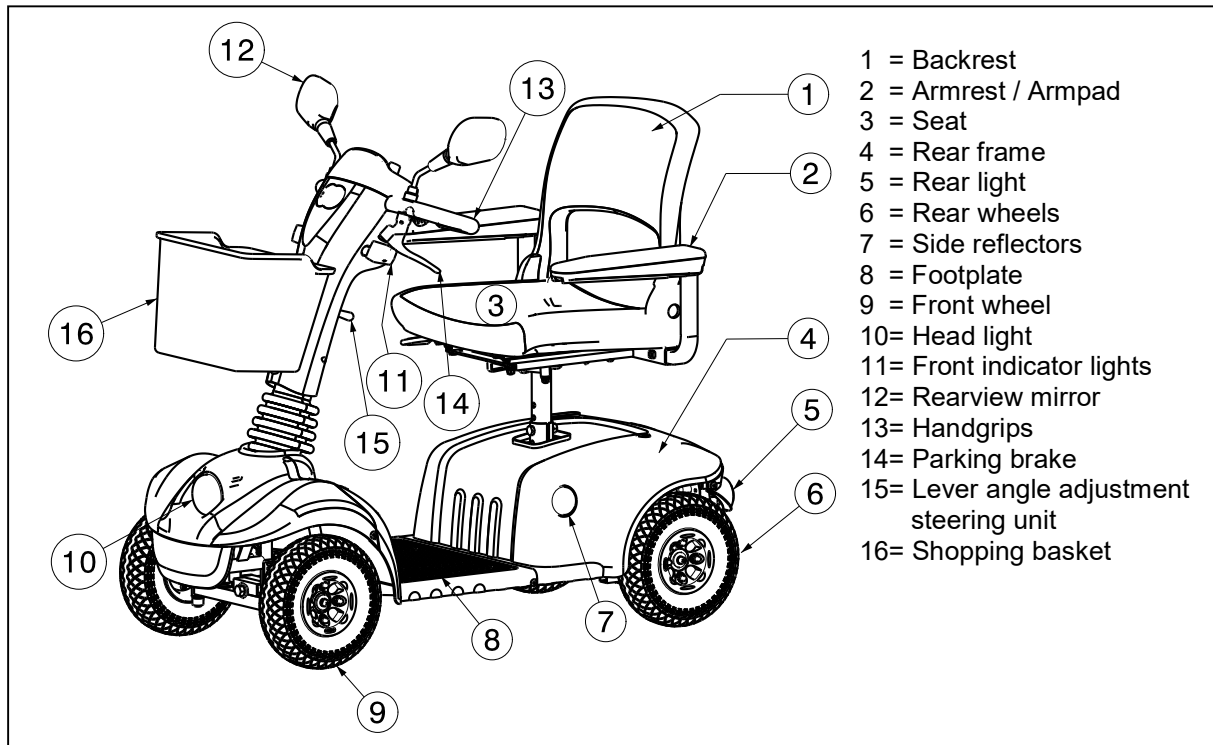

1.6 Location identification plate

The location of the identification plate ① is on the rear cover. You can also find an identification plate under the battery cover.

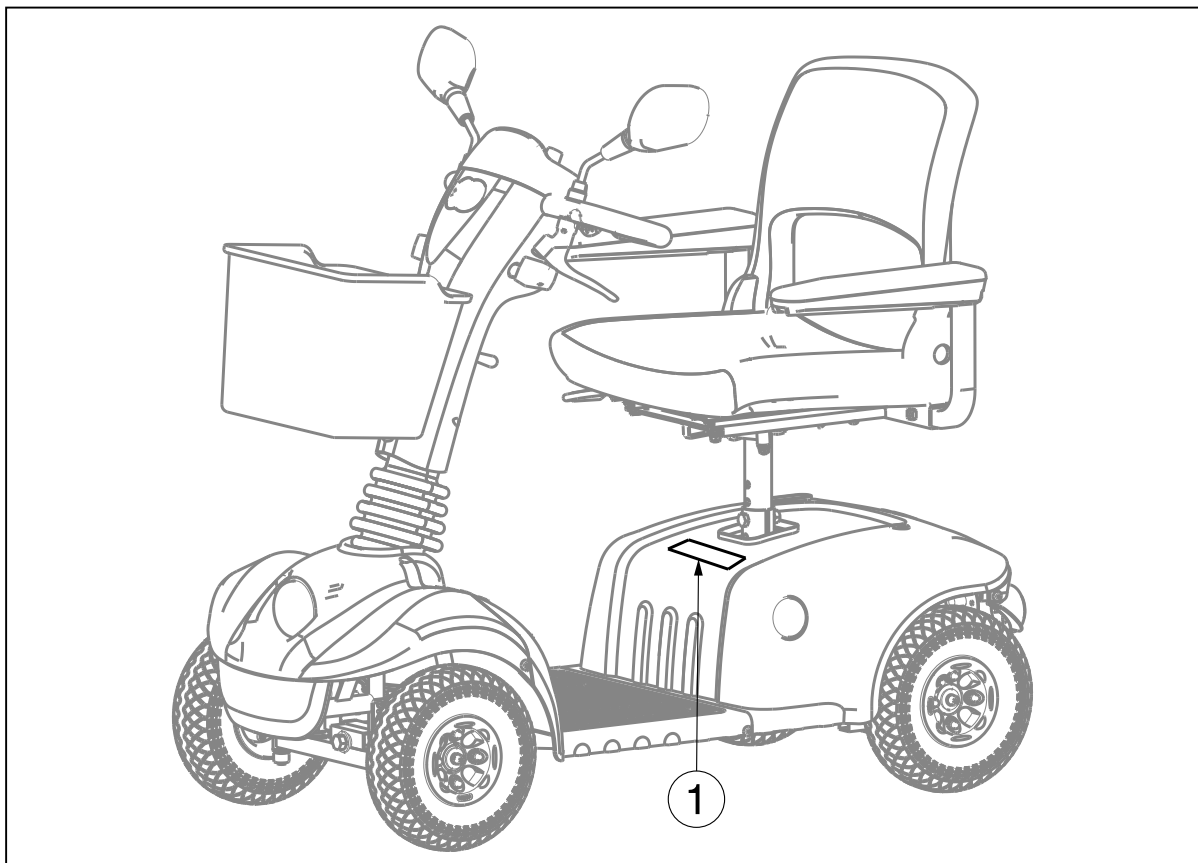1.7 Explanation of symbols

Maximum mass

Outdoor use

Indoor use (only for battery charger)

Separate recovery and recycling of electric and electronic devices (only for battery charger)

Protection class II

Maximum safe slope

CE conformity

Maximum speed

Not intended to be used as a seat in a Motor vehicle

Type designation

Warning: Do not Pinch hands / fingers

1.8 Expected lifespan

The average lifespan of your scooter is 5 years. Depending on the frequency of use, driving circumstances and maintenance, the lifespan of your scooter will increase or decrease.

2 Use

This chapter describes the everyday use. **These instructions are for the user and the specialist dealer.**

The scooter is delivered fully assembled by your specialist dealer. The instructions intended for the specialist dealer on how to set up the scooter are given in § 3.

2.1 Remarks on electromagnetic compatibility (EMC)

Your scooter has been tested according ISO 7176-21.

We wish to point out that sources of electromagnetic waves (e.g. cell phones) are liable to create interferences. The electronics of the scooter itself could affect other electric appliances too.

To reduce the effect of electromagnetic sources of interference, please read the following warnings:

- ⚠ WARNING: The scooter might disturb the operation of devices in its environment that emit electromagnetic field.**
- ⚠ WARNING: The driving performance of the scooter can be influenced by electromagnetic fields (e.g. electricity generators or high power sources).**
- ⚠ WARNING: Avoid using any portable TV or radio in the immediate environment of your scooter for as long as it is turned on.**
- ⚠ WARNING: Avoid using any transmitter-receiver or cell phones in the immediate environment of your scooter for as long as it is turned on.**
- ⚠ WARNING: Check the area for transmitter masts and avoid using the scooter close to them.**
- ⚠ WARNING: If involuntary movements or braking occur, turn off the scooter as soon as it is safe to do so.**

Interfering electromagnetic fields may have a negative effect on the scooter's electronic systems. These can include:

- Disengagement of the engine brake
- Uncontrollable behavior by the scooter
- Unintentional steering movements

In the presence of very strong or enduring interfering fields, the electronic systems could even break down entirely or suffer permanent damage.

Possible sources of radiation include:

- Portable receiver and transmitter installations (receiver and transmitter with fixed antenna mounted directly on the transmitting unit)
 - Transmitting and receiving sets
 - Cell phones or "walkie talkie"
 - portable TV, radio and navigation devices
 - other personal transmitting devices
- Mobile medium-range transmitting and receiving devices (e.g. car antennas, with the antenna mounted on the outside of the vehicle)
 - Fixed transmitting and receiving sets
 - Fixed mobile transmitting and receiving devices
 - Fixed radio, TV and navigation systems
- Long-range transmitting and receiving devices
 - Radio and television towers
 - Amateur radio sets

Other home devices like CD player, notebook, cordless phones, AM/FM radios, electric shavers and hair dryers, so far as we know, will have no influence provided that they function perfectly and their cabling be in an excellent condition. Please conform to the operating instructions accompanying such electric instruments to ensure trouble free operating of your scooter.

1. Do not operate hand-held (transceivers-receivers), such as citizens band (CB) radios, or turn ON personal communication devices, such as cellular phones, while the powered vehicle is turned ON;
2. Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them;
3. If unintended movement or brake release occurs, turn the vehicle OFF as soon as it is safe;
4. Be aware that adding accessories or components, or modifying the powered vehicle, may make it more susceptible to EMI (Note: There is no easy way to evaluate their effect on the overall immunity of the powered vehicle);

The intensity of the interfering EM energy can be measured in volts per meter (V/m). Each powered vehicle can resist EMI up to a certain intensity. This is called its "immunity level". The higher the immunity level, the greater the protection. At this time, current technology is capable of achieving at least a 20 V/m immunity level, which would provide useful protection from the more common sources of radiated EMI. This powered vehicle model as shipped, with no further modification, has an immunity level of 20 V/m without any accessories.

2.2 Carrying the scooter

The best way to carry the scooter is to make use of neutral mode of the scooter. Place the scooter in neutral and roll the scooter to the desired place.

If this is not possible you can carry the scooter by following steps:


1. Switch the scooter off.
2. Remove loose parts (seat and batteries).
3. Store loose parts in a safe place.
4. Carry the frame + steering unit with 2 or 3 persons to the desired place. Take the frame on the chassis, and not by the bumpers or the plastic parts. Only on the fixed parts from the frame.

2.3 Assembly and disassembly of the scooter

- ⚠ **WARNING:** Risk of injury - Make sure that all movable parts are assembled properly.
- ⚠ **WARNING:** Risk of injury - Assembly and disassembly must be done by trained personnel (e.g. specialist dealer).
- ⚠ **WARNING:** Risk of injury - Be careful not to cut or bruise yourself with the cables.
- ⚠ **WARNING:** Risk of injury - Turn the scooter off before disassembly.
- ⚠ **CAUTION:** Risk of pinching - Do not place fingers between the components of the scooter.


To disassemble, please proceed as follows:

- Switch the scooter off.
- Remove the seat (see "Adjusting the seat" chapter).
- Lift off the rear plastic cover of the batteries.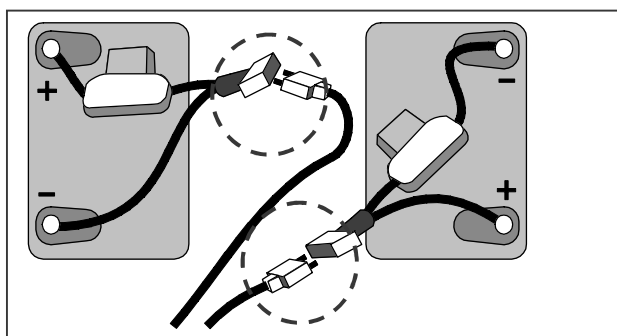

- Loosen the straps used to fasten the batteries in place.
- Unplug all battery plugs (not the pole connectors) and all cable plug- and socket connections.
- Lift the batteries out.
- Fold the steering unit down.

To reassemble, please proceed as follows: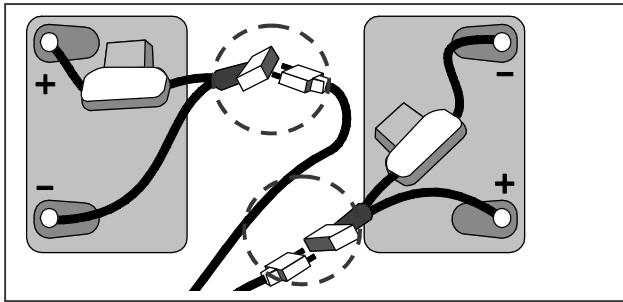


- Move the steering unit upwards.
- Put the batteries back in and connect to the battery plugs (plugs of the same color go together).
- Use the straps to tighten the batteries in such a way that they will not move, even during drives.
- Mount the rear plastic cover.
- Mount the seat (see "Adjusting the seat" chapter).

2.4 Operating the brakes

To apply the electromagnetic brakes:

1. Let go of the speed or drive lever, the electromagnetic brake in the motor will be activated, the scooter will stop.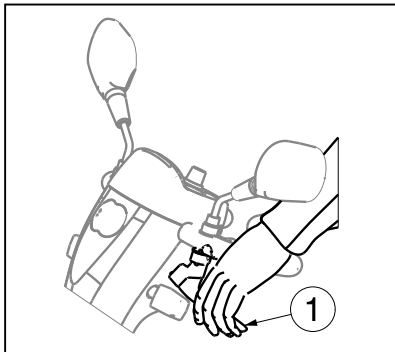


To apply the parking brakes (optional):

1. Pull the brake lever ① towards the handgrip.
2. Release the brake lever ①.

The parking brake tightness can be adjusted by the brake cable adjuster.

2.5 Transfer in and out the scooter

⚠ WARNING: Risk of injuries – Always turn your scooter OFF before getting on or off your scooter.

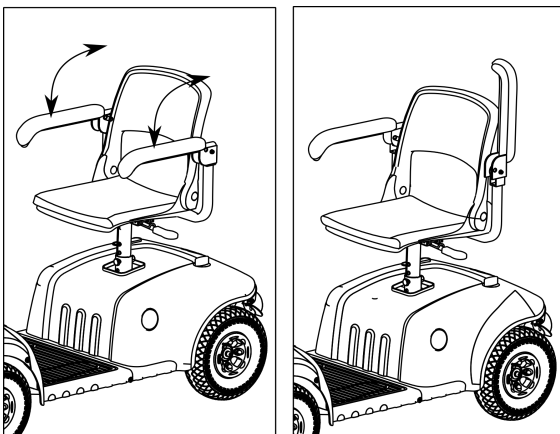1. Park the scooter as close as possible to the place where you want to transfer.
2. Check that the control panel (scooter) is switched off.
3. Put the armpad upwards on the side you want to transfer.
4. Transfer to / from the seat of the scooter.

2.6 Correct position in the scooter

Some recommendations for a comfortable use of the scooter:

1. Position your backside as close as possible to the backrest.
2. Make sure your upper legs are horizontal.

2.7 Driving the scooter

- ⚠ WARNING: Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.**
- ⚠ WARNING: Risk of unsafe settings - Use only the driving characteristics described in this manual.**
- ⚠ WARNING: Risk of injuries – Always turn your scooter OFF before getting on or off your scooter.**

2.7.1 Preparing the scooter for use

When you use your scooter for the first time, make sure that it is standing on level ground. All wheels must be in contact with the ground.

1. For first time use, fully recharge the batteries.
2. Make sure that the motor is engaged.
3. Set the steering unit in the most comfortable position for yourself and make sure it has been secured properly.
4. See that the seat has been locked in the driving position.
5. Sit down on the seat and verify that both armrests are folded down so you can rest your arms on them.
6. Insert contact key and turn to the right, please wait for at least 3 seconds before you press the speed lever otherwise you have a “delay protection” alarm. If necessary, turn on the headlights.
7. Check the battery indicator and see that there is enough power for your trip. If not enough capacity recharge the batteries before departure.

Now put the speed control on the operator control to the minimum position. Your scooter is now ready for use. Only use higher speed setting when you are confident that you can easily operate and control your scooter.

Before using the scooter in busy or potentially dangerous areas, familiarize yourself with the operation of your scooter. Please practice in a wide and open area like a park.

2.7.2 Handling after usage

Before getting off from your scooter, make sure that all four wheels touch the ground simultaneously. Next turn the key to the “OFF” (charge indicator goes out) to turn off the integrated light.

2.7.3 Parking the scooter

Once your scooter has been turned off, no command can be sent to the driving system. You will be unable to deactivate the electromagnetic brakes before turning on your scooter again. Always park your scooter on sites that are well guarded or clearly visible.

2.7.4 Your first trip

- ⚠ WARNING: Control your scooter - Accustom yourself with the driving behavior of your scooter.**
- ⚠ WARNING: Control your scooter - Do not use both sides of the speed lever simultaneously. This might leave you unable to control your scooter.**
- ⚠ WARNING: Control your scooter, risk of injury - Do not adjust the speed while you are driving.**
- ⚠ WARNING: Risk of injury - Be sure no people or objects are behind you while reversing the scooter.**

- Driving

Once you are seated on your scooter and have started it as described above, take hold of the handgrips with both hands, put your fingers to pull the speed lever in the required direction, i.e.:

PULL ACTION BY THE RIGHT HAND	=	FORWARD MOVEMENT
PULL ACTION BY THE LEFT HAND	=	BACKWARD MOVEMENT

While using your scooter indoor, do not put your scooter in the highest speed.

- Driving on the pavement

Ensure the limit switch is set to the slowest setting when using the scooter on the pavement. The maximum speed to drive on the pavement depends from country to country. Check the traffic law from the relevant country.

The limit switch can be set to the fastest setting for use on the road or on private ground.

- Braking

To brake, let go of the speed/drive lever, which will let it return to the zero position and slow down your scooter to a gentle stop. Practice pulling away and braking to get accustomed to the scooter. You need to be able to estimate how your scooter will react when you drive or brake.

Do not turn your scooter "OFF" while driving as this will lead to an emergency stop and possible risk of accident and injury.

- Driving in corners and bends

⚠ WARNING: Risk of tipping over - Reduce your speed before you enter a curve or corner.

⚠ WARNING: Risk of clamping - Always maintain an adequate distance from corners and obstacles.

⚠ WARNING: Risk of tipping over - Do not drive your vehicle in an "S" pattern or make erratic turnings.

For corners and bends, use both hands to turn the handle bar in the direction you wish to take. The front wheels will turn accordingly and steer the scooter into a new direction. It is very important that you ensure that there is enough space allowing you to go around bends and corners. Narrow passages must preferably be approached in a large curve so as to allow you to enter the narrowest part as straight from the front as possible. Bear in mind that your scooter's back part will in most cases be wider than the front part.

Take care not to enter curves and corners diagonally. By 'cutting the corner', chances are that your rear wheels will run into obstacles and destabilize your scooter.

2.7.5 Backward movement

⚠ WARNING: Control your scooter - Accustom yourself with the driving behavior of your scooter.

⚠ WARNING: Control your speed - Always go backward in the lowest speed possible.

⚠ WARNING: Risk of collision - Always look to the back when you go backward.

Rearward driving requires increased concentration and care (LEFT HAND ACTION). This explains why we have greatly reduced the speed of rearward movement in comparison with forward driving. But we still recommend that you put the speed lever on minimum when you drive backward.

Do not forget that the steering direction with rearward driving is the opposite of when you go forward, and that your scooter will turn directly in the required direction.

2.7.6 Going uphill

⚠ WARNING: Control your scooter - Accustom yourself with the driving behavior of your scooter.

⚠ WARNING: Control your scooter - Never put your scooter in neutral on slopes.

⚠ WARNING: Control your speed - Moving on slopes as fast as possible.

⚠ WARNING: Risk of tipping over - Do not exceed the maximum gradient of static stability uphill (see paragraph "Technical specifications").

⚠ WARNING: Never reverse going uphill.

Sitting position, going uphill

- Adjust your seat depth and backrest angle, in order to sit in most front position, see section §3.4 and §3.5 for adjustment.
- Always lean forward when climbing a steep gradient.

Always approach slopes directly from the front and, to avoid tipping over, see that all four wheels stay in contact with the ground at all times (ramps, driveways, etc.). Your scooter is powered through a differential. Both drive wheels should therefore stay in contact with the ground at all times. Should one of the drive wheels no longer be in contact with the ground, a safety device will cut the transmission of energy to the wheels, bringing the scooter to a halt.

If you come to a stop on a slope by letting go of the accelerator, the engine break will prevent your scooter from rolling backward. As soon as the accelerator returns to zero position, the engine break is activated.

To resume your uphill drive, push the accelerator throttle open fully to ensure the release of a sufficient amount of power. This will allow your scooter to slowly ascend the slope.

If your scooter is unable to drive up, turn the speed control up and try again.

When driving on a gradient, the battery indicator light might move up and down. This is a normal phenomenon so please do not worry.

2.7.7 Going downhill

- ⚠ WARNING: Control your scooter - Accustom yourself with the driving behavior of your scooter.**
- ⚠ WARNING: Control your scooter - Never put your scooter in neutral on slopes.**
- ⚠ WARNING: Control your speed - Moving on slopes as slow as possible.**
- ⚠ WARNING: Risk of tipping over - Avoid sharp bends.**
- ⚠ WARNING: Risk of tipping over - Do not exceed the maximum gradient of static stability downhill (see paragraph "Technical specifications").**

Sitting position, going downhill

- Adjust your seat depth and backrest angle, in order to sit in most rearward position, see section §3.4 and §3.5 for adjustment.
- Always lean backward when descending a steep gradient.

Always approach downhill slopes directly from the front. Slantwise approaches can lead to some of the wheels no longer staying in contact with the ground (danger of tipping over). If one of the back wheels fails to make contact, the transmission of power will be cut and the scooter come to a standstill.

The weight of the scooter will increase your downhill speed. Turn the speed control down and adjust your speed to the conditions.

Avoid sharp bends on downhill slopes. The weight of your scooter could cause your scooter to lift up on one side or even fall over around bends.

2.8 Driving the scooter on ramps

- ⚠ WARNING: Risk of injury - Do not exceed the maximum load of the ramps.**
- ⚠ WARNING: Risk of injury - Choose the correct ramps so there cannot be caused any injury or damage.**
- ⚠ WARNING: Risk of injury - Note that the considerable weight of the electric scooter generated large reverse forces when an attendant is pushing the scooter over the ramps.**
- ⚠ WARNING: Risk of injury - Be sure that the wheel height is high enough to get over the free height of the ramps. The scooter frame may not touch the ramps.**

Should you wish to use ramps to drive over an obstacle, please note the following:

1. Find out from the manufacturer what the maximum load is for the ramps.
2. Drive on the ramps at the lowest speed possible.
3. See the instructions in the chapter "your first trip".

2.9 Driving the scooter over obstacles

⚠ WARNING: Risk of injuries by unexpected movement


- Do not use your scooter on an escalator.
- Do NOT drive on slopes, obstacles, steps or kerbs larger than the size described in the technical specification in §1.3.
- Approach a kerb always from the front.
- Make sure that all wheels make contact to the ground during driving up or down.

⚠ CAUTION: Risk of damage

If necessary, take a short run up to speed-up the scooter to mount the obstacle or kerb. Prevent that you or your scooter has to cope with a large recoil.

2.10 Operator control

- Put the ON/OFF key switch on, please wait for at least 3 seconds before you press the speed lever otherwise you have a "delay protection" alarm.
- Now the battery level indicator ⑧ will light up and shows the current level of your batteries.
- Now turn the speed switch ② to the desired driving speed.
- Pull the drive lever with your fingers backwards (right lever for forward movement, left lever for backward movement).
- To sound the horn, press the horn button ④.
- To put on the front and back lights, press the light button ③.
- To put on the emergency flashers, press button ⑥.
- To put on the indicator lights, press the desired button ⑤ and ⑦ (left = left indicator, right = right indicator).
- To stop the scooter loosen the speed lever under the operator control.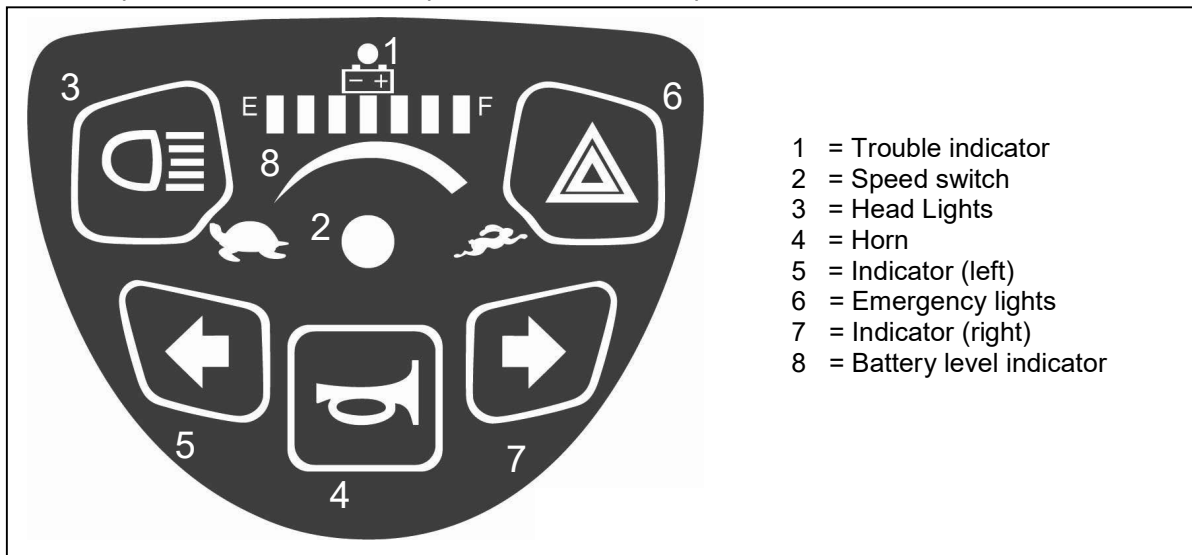

Vermeiren is responsible for the changes in the software. For changes in the software contact Vermeiren.


2.11 Neutral

- ⚠ **WARNING:** Control your scooter - Never put your scooter in neutral while you are driving.
- ⚠ **WARNING:** Control your scooter - Never put your scooter in neutral on slopes. It may accidentally roll away.
- ⚠ **WARNING:** Risk of injury - Never pick up the electronic drive before stopping the motor/gears to prevent motor from overheating.

The scooter is fitted with a freewheel device that is accessible and operable by the attendant or the user when not sitting in the scooter. You should only use the scooter in neutral to transport the scooter or to move it out of a danger zone.

Put the scooter in neutral with the motor stop lever ① on the back side of the scooter.

- Driving
 1. Put the motor stop lever ① on drive. This will again interlock the motor and gears.
 2. Switch the ON/OFF key ON.
 3. Electronically controlled driving is now possible.
- Neutral
 1. Switch the ON/OFF key OFF.
 2. Put the motor stop lever ① on neutral (see label). This separates the motor from the gears.
 3. The scooter can now be pushed without electronic drive.

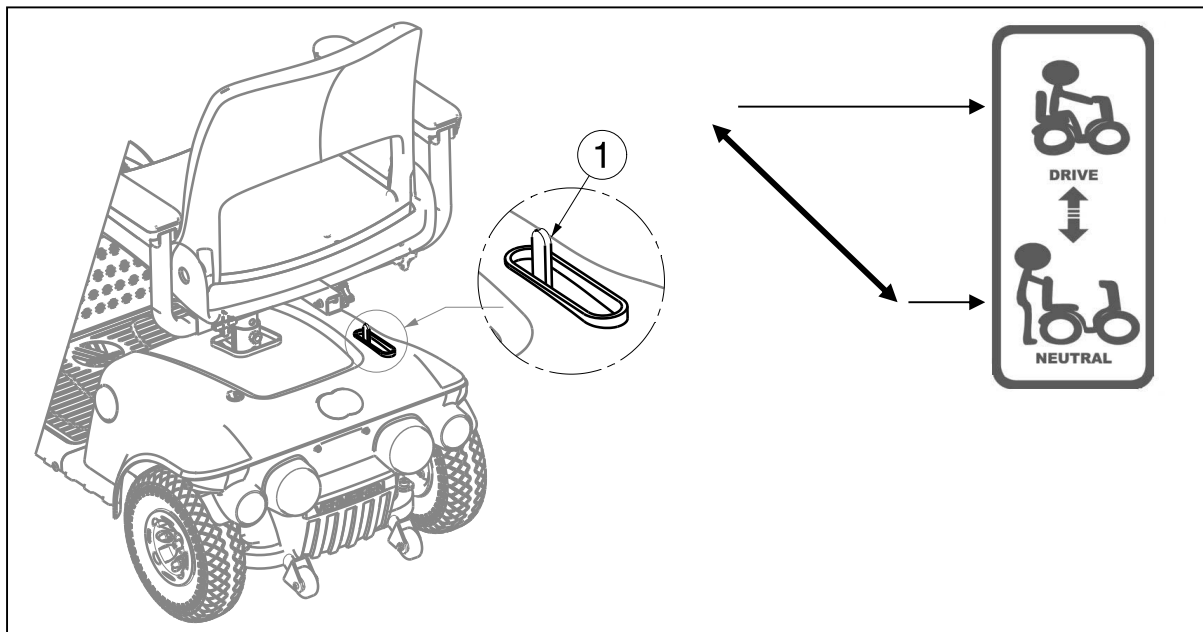2.12 Transport

2.12.1 By aircraft

i Since the scooter is equipped with non-spillable AGM battery, it is possible to transport the scooter (as a whole) by aircraft. Consult your airline for the correct handling procedures before booking your flight.

Before handing your scooter over to the airport personnel, disconnect, and if necessary, remove the battery from the scooter. See also section §3.11 about the battery replacement.

2.12.2 By car

-  **DANGER:** Risk of injury - The scooter is not suited for use as a seat in a motor vehicle as indicated by the symbol in the following picture.
-  **WARNING:** Risk of injury - Remove all loose parts prior to transportation.
-  **WARNING:** Risk of injury - No people or objects should be under the scooter, on the footplate or seat during transportation.
-  **WARNING:** Risk of injury - See that the scooter is attached properly. So you can avoid injury of passengers during collision or sudden braking.
-  **WARNING:** Chance of pinching - Do not place fingers between the components of the scooter.

The best way to transport your scooter in the car is to drive the scooter in the car by using ramps. When you are not experienced to drive the scooter by using of ramps you can also put the scooter in neutral mode and push the scooter in the car by using ramps.

When the scooter does not fit in the car it is also possible to transport the scooter by the following steps:

1. Remove all loose parts prior to transporting (seat, etc.).
2. Store loose parts safely.
3. If possible, remove the batteries / battery boxes to save weight.
4. Fold the steering unit down with the angle adjustment.
5. Place the scooter in the car by 2 or 3 persons.
6. Attach the frame of the scooter securely to the vehicle.

2.13 Automatic power shut down

In order to avoid accidental battery run down, your scooter is equipped with an automatic power shut down facility. If the scooter is switched on, after remaining undisturbed for a period it will automatically turn off. Should this occur, simply switch your scooter off and back on and it will be ready to use once again.

2.14 Charging the batteries

-  **WARNING** Risk of injuries and damage due to fire
 - Only use the battery/batteries and charger supplied with your product. The use of any other battery or charger may be dangerous (fire hazard). Contact your specialist dealer for possible replacement.
 - Under no circumstances open or change the battery, charger, cables, plugs nor change any connection points!
 - The charger is only intended to charge the battery/batteries delivered with the scooter, not to charge any other batteries.
 - Protect the battery and battery charger against flames, high temperatures (>50°C), moisture, sunshine, severe shocks (for example dropping). Do NOT use the battery if this has been the case.
 - Charge the battery with the supplied charger, indoors, in a well-ventilated area, out of children's reach.
 - Do not use any extension cord for charging.

-  **CAUTION** Risk of injuries

The batteries will last shorter or longer depending on the use, terrain and driving conditions. When you plan a trip, take the battery-charge status in to account. Be aware of the action radius of the battery, see the technical details in §1.3.

CAUTION Risk of battery damage, reduction of battery life span


- Read the charger instruction manual and observe the safety guidelines regarding the charger.

- If your scooter is stored for a while, it is advised to charge the battery regularly (approximately once a month).
- Do not interrupt the charging cycle; wait till the charger indicates that the battery is fully charged, see the charger instruction manual. Prevent charging for a short period. Note: this is NOT applicable for Lithium batteries.
- The battery may be irreversibly damaged if the battery is fully discharged. Therefore, make sure that the battery is charged on time.
- The lifespan reduces if the battery is exposed to extreme cold for a longer period of time, see the technical details in §1.3.
- Make sure that all connection points on the battery and the connection point in the battery housing are free of dust and other contamination.
- Protect your battery from large sources of electromagnetic radiation.

Fully recharge your new battery before its first time use.

The light emitting diode (LEDs) in the steering unit indicates the remaining capacity of your batteries. If the battery charger is turned on and no LED's are lit, check the fuse. If the red LED does not illuminate the charger is defective, consult your specialist dealer.

1. Switch the scooter OFF and remove the key.
2. FIRST connect the battery charger to a 230V AC power socket.
3. If applicable, open the cover (2) of the charging point on the scooter, see picture.
4. Connect the charger.
5. Refer to the battery-charger manual for the explication of the indicators on the charger.
6. When the battery is fully loaded, disconnect the battery.
7. Switch the scooter ON and check if all lights of the battery status indicator are ON.
8. Remove the charger from the socket, let it cool down and store it on a dry location.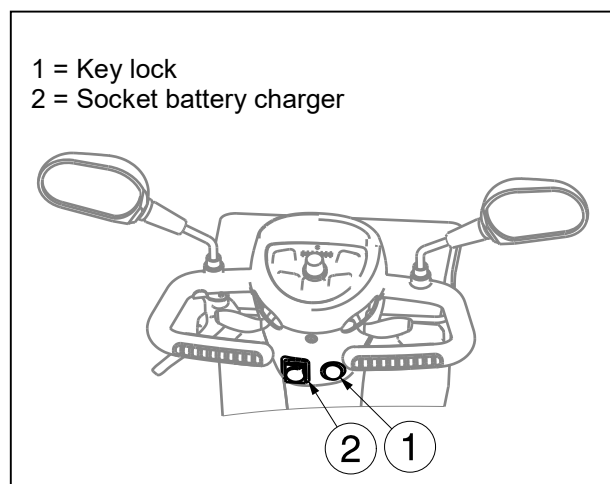

i In case of any problems whereby the battery cannot be charged according to the instructions in the charger manual, please contact your specialist dealer.

When the operating console displays the error code "Drive inhibit" after charging, make sure that the battery charger is disconnected. This safety measure prevents driving with the charger still connected to the supply grid.

3 Installation and adjustment

The instructions in this chapter are for the user and the specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

- ⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.**
- ⚠ WARNING: Risk of tipping over - Variation allowed adjustments can still change the stability of your scooter (tilt back or sideways).**

3.1 Tools

To set up the scooter the following tools are needed:

- Wrench set n° 13, 17
- Allen keyset n° 6

3.2 Manner of delivery

The scooter shall be delivered with:


- Frame with armrests, steering unit, rear and front wheels
- Footplate
- Seat + backrest
- Batteries (2x), motor
- Basket
- Rear view mirrors (2x)
- Parking brake
- Battery charger
- Accessories
- Manual

3.3 Adjusting the steering unit

The steering unit can be adjusted in many different positions (stepless) to suit each driver. The tiller angle (A) is adjustable between 0° (horizontal, without seat) and 80° (top position).

- ⚠ WARNING: Risk of injury - Never adjust the angle while you are driving.**
- ⚠ WARNING: Risk of injury - Turn the scooter off before you adjust the angle of the steering unit.**
- ⚠ WARNING: Risk of injury - Do not lean with your whole body on the steering unit.**

1. Push the lever ① downwards.
2. Adjust the steering unit in the desired position.
3. Release the lever ①.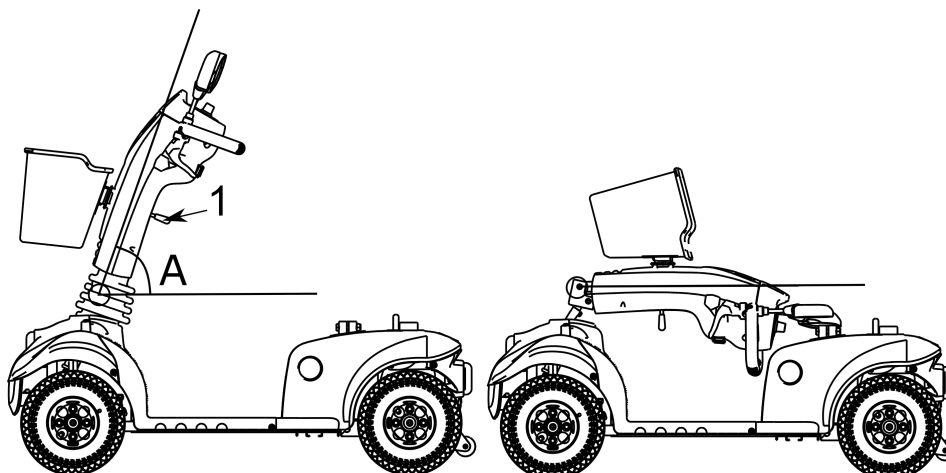

3.4 Adjusting the seat

⚠ WARNING: Risk of injury - Never perform adjustments while you are driving.

⚠ WARNING: Risk of injury - Make sure that the seat is locked securely.

Remove seat (Fig. A)

1. Pull the seat lever ① upwards.
2. Turn the seat ② a little bit and lift the seat out the adjustment bar ⑤.
3. Release the seat lever ①.

Lock the seat in place (Fig. A)

1. Pull the seat lever ① upwards.
2. Place the seat ② on the adjustment bar ⑤ and at the same time keep the seat lever ① pressed in.
3. Release the seat lever ①.
4. Check that the seat is firmly secured.

Swivel seat (Fig. A)

1. Pull the seat lever ① upwards.
2. Turn the seat ② in the desired direction.
3. Release the seat lever ①, and the seat will always stop after 90°.
4. Check that the seat is firmly secured.

Depth adjustment (Fig. A)

1. Pull the seat depth adjustment lever ③ outwards.
2. Move the seat ② forward or backward over the seat depth rails ④.
3. To lock the seat in place, let go of the lever ③ once the seat ② has reached the desired position.
4. Swivel the seat a little until it locks in position.
5. Check the seat is locked securely.


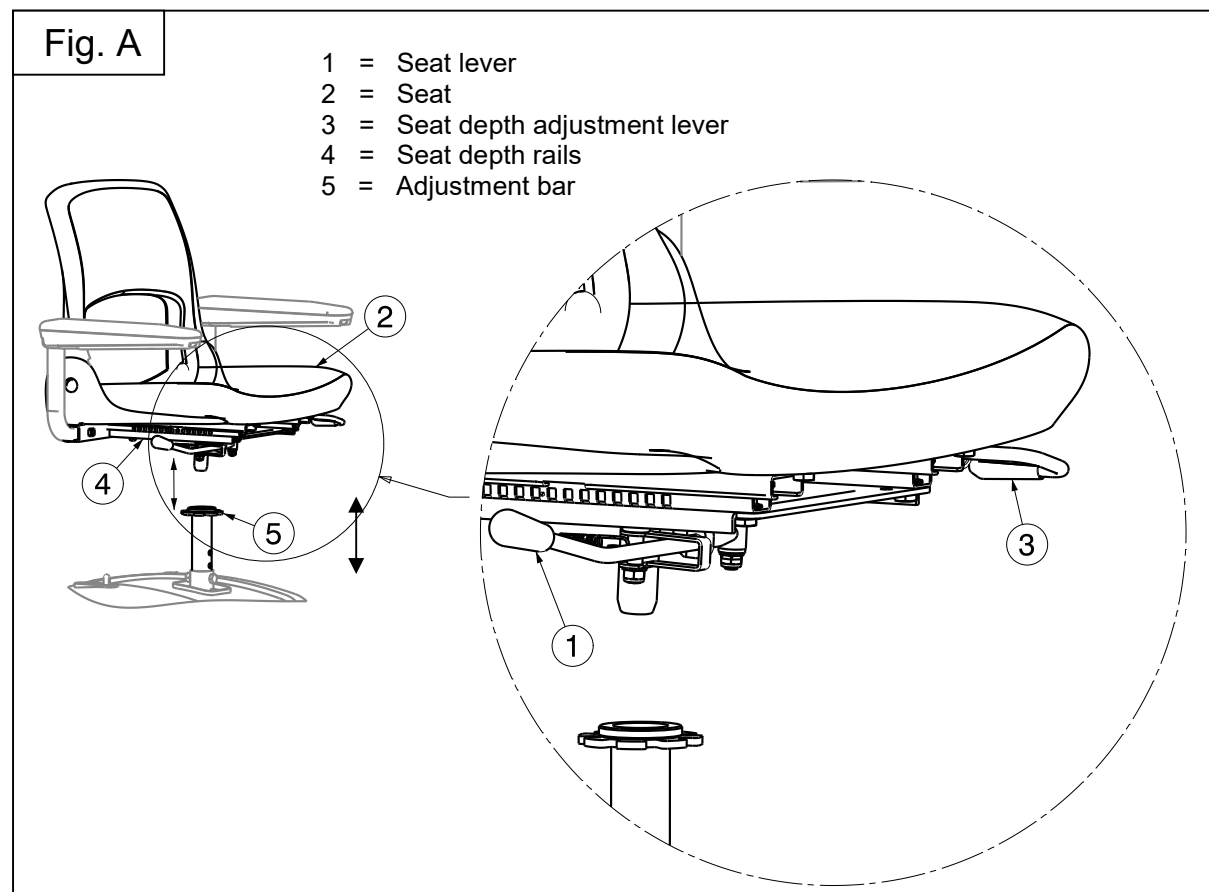
Fig. B

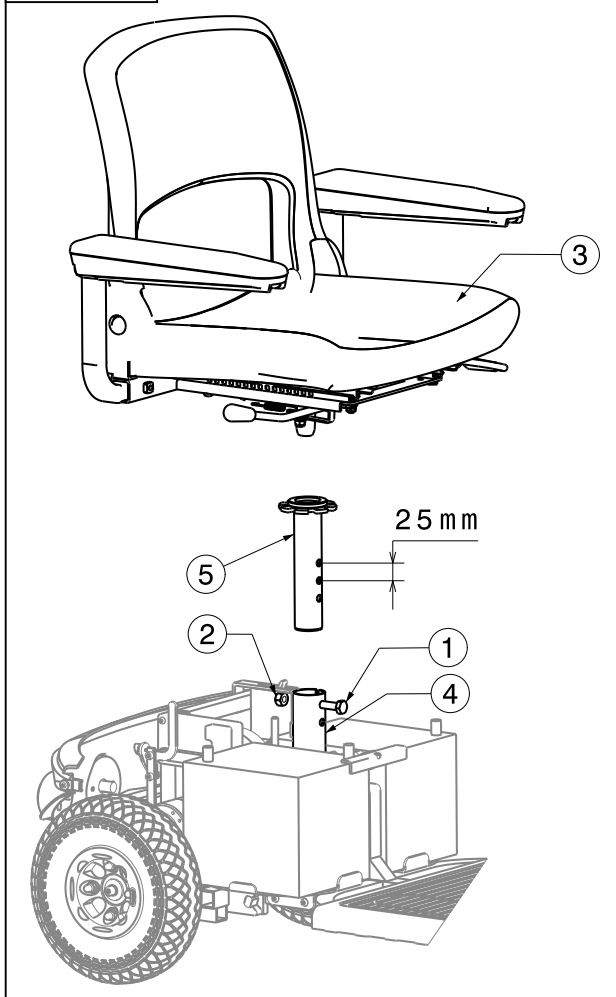
Seat height adjustment (Fig. B)

The seat can be adjusted in 3 different seat heights (increments: 25 mm).

1. Remove the seat ③ and battery cover.
2. Remove the bolt ① and nut ②.
3. Move the adjustment bar ⑤ upwards/downwards in the receiver ④, and place this in a comfortable seat height.
4. Fasten the bolt ① and nut ②.
5. Replace the seat ③ and battery cover.
6. Check the seat ③ is locked securely.

3.5 Folding the backrest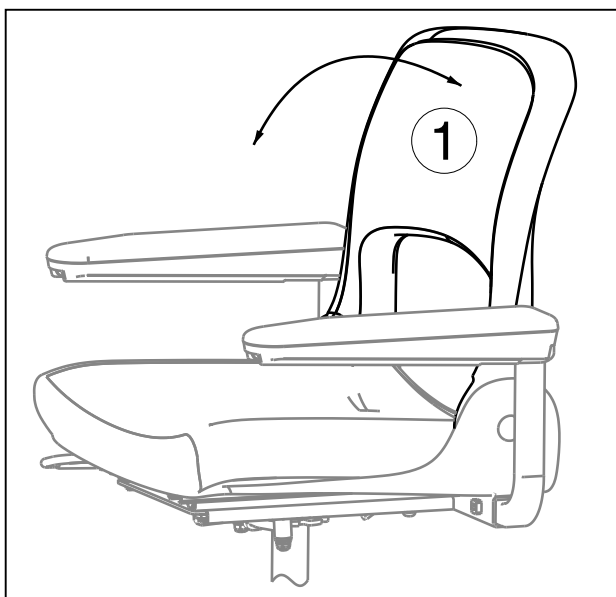


⚠ WARNING: Risk of injury - Never perform adjustments while you are driving.

Push the backrest ① forward to fold the backrest to the front.
Pull the backrest ① backwards to place the backrest in the upright position.

3.6 Adjusting the rear view mirrors

It is possible to adjust the rear view mirrors: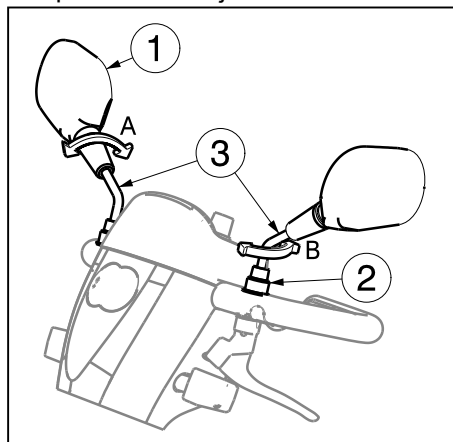

Adjustment 1 (arrow A):

1. Grasp the rear view mirror ①.
2. Rotate the rear view mirror ① in the desired position.

Or

Adjustment 2 (arrow B):

1. Move the cap ② upwards.
2. Loosen the mirror screw.
3. Grasp the tube ③ gently.
4. Rotate the tube ③ and put the mirror ① in the desired position.


3.7 Adjusting the armrests

⚠ CAUTION: Risk of tipping over - Make sure that the armrests are positioned symmetrically from the seat.

⚠ CAUTION: Risk of injury - Mount the inner tube always with of safe distance of 50 mm in the outer tube.

The armrest can be adjusted in width.

3.7.1 Adjusting width armrests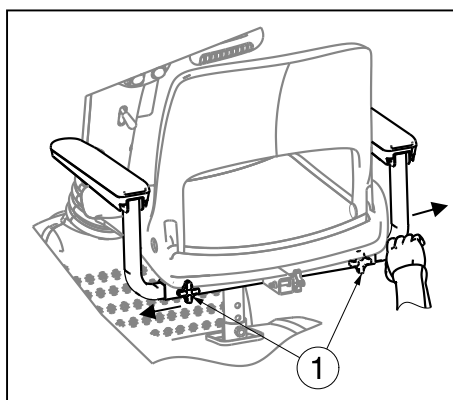

1. Remove the star knob ① on the back of the scooter.
2. Grasp the inner square tube as shown in the picture.
3. Move these tube to adjust the armrest to the desired width (range 100 mm on each side).
4. Retighten the star knob ①.
5. Check that the armrest is fixated very well.

3.8 Tyre changing

⚠ CAUTION: There must be no air in the tube before it can be removed.

⚠ CAUTION: Risk of damage - If handled improperly, the rim might be damaged.

Note the following before inserting the new tube:


Check the rim bed and the inside wall of the tyre for foreign matter and clean these properly if necessary. Check the condition of the rim bed, especially around the position of the air valve. Please use only genuine original replacement parts. No liability is accepted for damage caused by non-genuine replacement parts. Kindly contact your specialist dealer.

Assembly:

⚠ WARNING: Risk of injury - Check that the pressure is correct.

⚠ CAUTION: Risk of injury - Make sure that no objects or body parts are pinched between the tyre and the rim when mounting a tyre.

⚠ WARNING: Risk of injury - Make sure all screws are firmly secured by hand before driving with your scooter. Screw adhesive (ex. Loctite) should be applied to the screws on the flange. Screw adhesive will only work if the thread is free of grease and particles.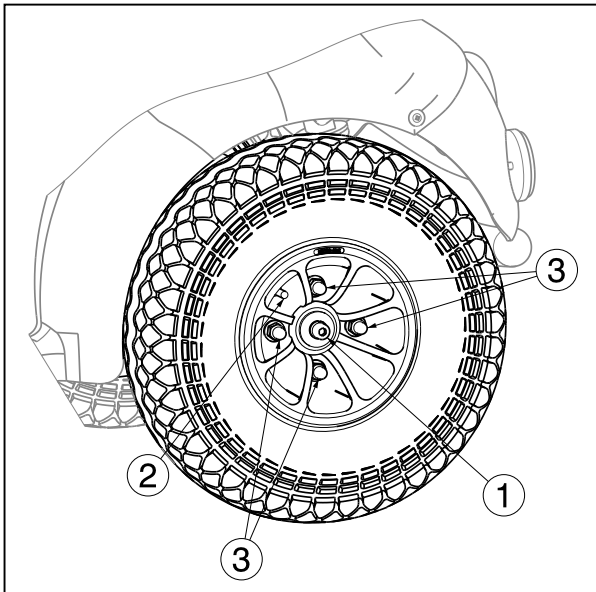

DISMANTLING

1. Unscrew and remove the middle screw ① that attach the wheel and flange to the scooter.
2. Let the air out of the wheel by lightly pressing the pressure pin on the valve ②.
3. Unscrew the 4 screws ③ on the inside of the rim. Separate the rim sides.

ASSEMBLY

Insert the partly-filled inner tube into the tyre.

1. Connect the two sides of the rim through the tyres and place the flange against the tyre and screw them back again with the 4 screws ③.
2. Put the valve ② through the hole for it in the rim.
3. Put the wheel and flange back on the scooter and secure this connection with the middle screw ①. Inflate the wheel to the recommended tyre pressure.

Check all around on both sides that the tube is not pinched between the rim and the edge of the tyre. Lightly push the air valve inwards and pull it out again to make sure that the tyre is positioned properly in the region of the air valve.


To ensure that the wheel is inflated correctly, admit only so much air initially that the tyre can still be easily pushed inwards by using your thumbs. If the check-lines are equidistant from the edge of the rim on both sides of the tyre, then the tyre is centered properly. If not - let out the air and position the tyre afresh. Now the tyre can be inflated to its full operating pressure (note the maximum) and the valve cap should be replaced.

Only an expert can guarantee correct assembly. Work not done by your specialist dealer, would void any warranty claims.

When inflating the tyres, always check that the pressure is correct. The correct pressure is given on the tyre walls.

Use only inflating equipment which complies with regulations and indicates the pressure in bar. We do not accept any liability for damage caused by using inflation equipment not supplied by the manufacturer.

3.9 Thermal fuses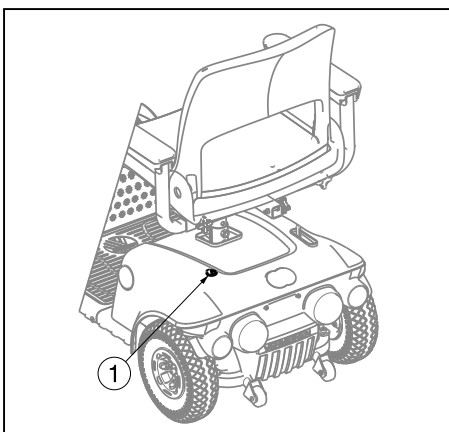


To protect the motor against overload, your scooter have a thermal safety mechanism ① on the rear of your scooter. The thermal fuse will automatically cut the motor to prevent overheating and thus rapid wear and tear or breakdowns. You can access the thermal safety device through a gap in the back of the synthetic cover.

The thermal safety device can go loose if you go up or down slopes that exceed the maximum gradient indicated. Nominal loads that exceed the maximum permitted could also cause the device to jump out. Likewise, if you keep on driving with the engine brake on, it could result in overload. The maximum values not to exceed are indicated in the chapter "Technical Specifications" of the corresponding manual.

To be able to use the scooter again, remove the overload and wait till the motor has cooled off. Then gently switch the thermal safety mechanism back in. Your scooter is now ready for use again.

3.10 Anti-tipping

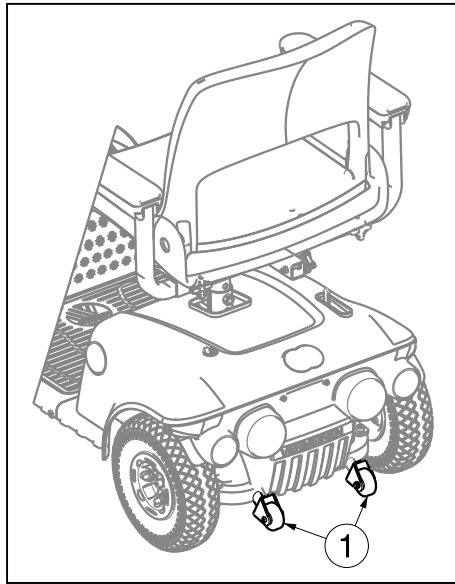An anti-tipping ① is fixed to the frame of the scooter on the back side. It is not possible to remove this. The anti-tipping is there for your safety. It prevents the scooter from tipping over backward when you drive over small obstacles that do NOT exceed the maximum height specified.

3.11 Changing the battery

- ⚠ **CAUTION:** Risk of burns - Do not come in contact with the acid from the batteries. See for a good ventilation of the battery compartment.
- ⚠ **CAUTION:** Risk of injury and damage – Only use Non-spillable batteries for replacement; these are indicated by a “Non spillable” mark on the battery.

Replace both batteries at once and not one battery.

Change the batteries of your scooter with the same type batteries (Type: **"AGM-Absorbed glass mat"** batteries).

The supplied battery charger works only with AGM batteries.

If you use other types of batteries, replace also the battery charger. Improper batteries, battery chargers voids any warranty claim. Contact your specialist dealer about other batteries, battery chargers in case of uncertainty.

The battery shall be changed by trained personnel.

The wiring and charger are well situated in the scooter. Do not attempt to re-locate the wiring by yourself. The improper installation of the wiring may result in pinching the wiring between the battery box, which cause failure to the electronic system of your scooter.

Be sure the battery cables are connected to the right battery.

4 Maintenance

For the maintenance manual of the scooters refer to the Vermeiren website:
<http://www.vermeiren.com/>.

CAUTION: Risk of injuries or damage

Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.

The service frequency depends on the frequency and intensity of use. Contact the dealer to agree to a common timetable for inspection/maintenance/repair.

Read the user instruction of the applied battery charger regarding maintenance.

4.1 Instructions for reuse

4.1.1 Product release for reuse

The last page of this manual contains a registration form for the specialist dealer to record each service. This page can also be used by the rental service to note the required inspection, cleaning, disinfection and maintenance. Check before each reuse if the maintenance is registered so that the product can be released for reuse.

4.1.2 Points of maintenance

Before each reuse, visually inspect the following points:

- A Battery status: Charge the battery, see §2.14.
- B Condition of wheels/tires, see §4.1.3.
- C All parts: Contamination, see §4.1.4.
- D Operating console and cables: Damage like wires that are frayed, broken or that become exposed.
- E Wheels, tiller, operating console: Well secured.
- F Seat and backrest: Dented spot, damages or tears. Replace the seat and/or backrest if necessary.

Contact your specialist dealer for possible repairs or part replacements.

Yearly or more often

Have your scooter inspected and serviced by your specialist dealer, at least once a year, or more often. The minimum maintenance frequency depends on use and should therefore be commonly agreed on with your specialist dealer.

If stored

Make sure that your scooter is stored dry to prevent mould to grow and damage the upholstery.

If your scooter is stored for a longer period, you still need to charge the batteries monthly. For more information, see §2.14.

4.1.3 Wheels and tires

- A Keep the wheels free of wires, hair, sand and carpet fibres.
- B Check the profile of the tyre. If the tread depth is less than 1 mm, the tyres need to be replaced. Contact your specialist dealer for this matter.
- C Check the tyre pressure of each tyre. The pressure depends on the type of tyre. The required pressure is readable on the tyre side.
- D If the tyre does not stay pressurised or is visibly flat, contact your specialist dealer for replacement.

CAUTION: Risk of injuries or damage

Make sure that the maximum pressure is not exceeded during inflation. Check the details on the side of the tyre.

4.1.4 Cleaning

CAUTION: Risk of damage by moisture

Keep the operating console clean and protect it from water and rain.

Never use a hose or high-pressure cleaner to clean the scooter.

Wipe all rigid parts of the scooter with a damp cloth (not drenched). If necessary, use a mild soap, suitable for varnishes and synthetics.

The upholstery can be cleaned with lukewarm water and a mild soap. Do not use abrasive cleaning agents to clean.

Keep the ventilation openings of the battery charger clean and free of accumulated dust. Blow away the dust and clean the battery charger enclosure with a slightly moistened cloth, if required.

4.1.5 Disinfection

⚠ CAUTION: Risk of damage - Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

5 End of use

At end of life, you need to dispose your scooter according to the local environmental legislation. The best way to do so, is to disassemble the scooter to facilitate the transport of recyclable parts. Usually, batteries are collected separately.

6 Declaration of conformity

This product is certified for CE and is classified as a Medical device, Class I.

The CE mark indicates the conformity of the product with standards for protection of health, safety and environment applied on products traded within the European economic area.

A copy of the Declaration of conformity is available via our website:

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Service registration form

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

Vermeiren GROUP
Vermeirenplein 1 / 15
B-2920 Kalmthout
BE

website: www.vermeiren.com

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All rights reserved, including translation.