Ninæe

casual

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ES - FAJA DE SOPORTE LUMBAR

EN - LUMBAR SUPPORT BAND

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CONTENTS / DESCRIPTION

- Ninaë Casual® lumbar support band:
- 1 removable and adaptable front part,
 - 1 posture reinforcement system with adjustable straps.

Product leaflet to read carefully.

INDICATIONS

During pregnancy:

Relieves lower back and pelvic pain. Reduces postural discomfort during activity. Alleviates the effect of gravity on the belly when standing.

After the birth:

Limits diastasis of the rectus abdominis muscle.

COMPOSITION

Monofilament:

41% polyamide, 38% elastomer coated with polyester, 21% polyester.

Reinforced fabric: 100% polyester.

Border:

64% polyamide, 36% elastomer.

Pelvic stays: Plastic (POM).

Dorsal stays: Steel.

Reinforcement straps:

76% polyamide, 24% elastomer.

Velcro: Polyamide.

SAFETY INSTRUCTIONS

This product must be fitted by a health professional (pharmacist, orthopaedist, physiotherapist, etc.). If rash, irritation or swelling remove the product and consult a doctor or prosthetist. Protect any part of the body to which an analgesic may have been applied before fitting the product. It is not recommended for other patients to reuse this product. In the event of a serious incident with the device, inform the manufacturer and the competent authority.

WASHING INSTRUCTIONS

- Wash by hand with neutral soap (max. 30 °C),
- Rinse carefully,
- Allow to dry in the open air,
- Do not dry in a dryer,
- Do not use bleach,
- Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sun light etc. During use or during washing do not use alcohols, ointments or dissolvent liquids. If the orthosis is not properly dried any detergent residues could irritate the skin and deteriorate the product.

Store at room temperature.

In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.

FITTING

A Figures 1 and 2

- Make sure the front part (Fig. 1) and the posture reinforcement system (Fig. 2) are the right way round using the Orliman Maternity® logos as a guide.

B Figure 3

- Fasten the front part to the posture reinforcement system, making sure the support band is centred at the pelvic and dorsal level.
- Adjust the tightness if necessary.

C Figure 4

Then fasten the two posture reinforcement side straps to the desired tightness.

D Figures 5 and 6

- Check the support band is correctly positioned (Fig. 5 and 6).
- The two side straps can be tightened during activity and loosened during periods of relaxation.

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