**KLY® LUBRICATING JELLY** (TR)**ÜRÜN ADI:**

KLY® KAYGANLAŞTIRICI JEL

**İÇERİK BİLGİSİ:**

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KLY® LUBRICATING JELLY

**COMPOSITION:**

Deionized water, Natural Glycerin, MonopropylenGlycol, Hydroxy Ethyl Cellulose, Methyl Hydroxybenzoate, Citric Acid

**INTENDED USE:**

This gel is used in endoscopy, colonoscopy, gynecological examinations, rectal endoscopic, catheter operations, also used for vaginal lubrication.

**INSTRUCTION FOR USE:**

**To Open The Tube:** Remove and reverse the cap then puncture the tube with it. Apply Lubricating Jelly to the gloves or the instrument in sufficient amount so that it forms a film over the area that comes in contact with patient. Wash off after use.

**To Open The Sachet:** Open carefully from marked area.

**SPECIAL PRECAUTIONS:**

**Precautions:** If container is damaged or opened, the Product is not guaranteed.

**CONTRAINDICATION:**

There is no known contraindication

**OVERDOSAGE AND SIDE EFFECTS:**

There is no known symptoms for overdose usage.

**Sensitivity To Glycerin:** Glycerin has emollient properties. Precautions Shall be taken when using the gel in people with sensitivity to glycerin.

**IDENTIFICATION:**

Transparent, water-soluble, greaseless and jelly.

**PRESENTATION:**

**Tubes Containing either:** 42 gr, 82 gr, 113,4 gr, 100gr or 250gr

**Sachets Containing either:** 2,5 gr, 3 gr, 3,5 gr, 4 gr, 4,5 gr, 5 gr, 10 gr

**STORAGE INSTRUCTIONS:**

Store between 5-30°C

**KEEP OUT OF REACH OF CHILDREN****NAME OF THE BRAND AND MANUFACTURER:**

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