
Monae

light

- FR - BANDE PELVIENNE
- ES - CINTURÓN PÉLVICO
- EN - PELVIC BELT
- DE - BECKENBAND
- PT - BANDA PÉLVICA
- PL - PAS MIEDNICOWY
- IT - FASCIA PELVICA
- EL - ΠΥΕΛΙΚΗ ΖΩΝΗ
- NL - BEKKENBAND
- SV - BÄCKENBAND
- RO - CENTURĂ PELVIANĂ
- LT - DUBENS JUOSTA
- RU - БАНДАЖ ТАЗОВЫЙ
- DA - BÆKKENBÆLTE
- JA - 骨盤ベルト
- AR- شريط الحوض

MISE EN PLACE / PÁSÄTTNING / POZIȚIONARE
SPOSÓB ZAKŁADANIA / UŽDĚJIMAS /
КАК НАДЕВАТЬ ИЗДЕЛИЕ / PLACERING
装着方法 / الوضع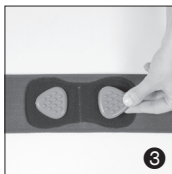

TU

TAILLE UNIQUE / TALLA ÚNICA / ONE SIZE

CONTENU/DESCRIPTION

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INDICATIONS

During pregnancy:

Relieves lower back pain, pubic symphysis pain (pubalgia), pelvic and/or sacroiliac pain. Protects the perineum.

Prevents and relieves sciatica pain.

After the birth:

Relieves pubic symphysis pain.

Supports the pelvic floor. Enables an additional action during perineal reeducation sessions.

(Can be used during daily activities and/or when standing for long periods).

COMPOSITION

Elastic band:

77% polyamide, 23% elastomer.

Fastening straps:

76% polyamide, 24% elastomer.

Velvet:

72% polyamide, 28% polyurethane.

Pads: Silicone.

Velcro: Polypropylene.

SAFETY INSTRUCTIONS

This product must be fitted by a health professional (pharmacist, orthopaedist, physiotherapist, etc.). If rash, irritation or swelling remove the product and consult a doctor or prosthetist. Protect any part of the body to which an analgesic may have been applied before fitting the product. It is not recommended for other patients to reuse this product. In the event of a serious incident with the device, inform the manufacturer and the competent authority.

WASHING INSTRUCTIONS

- Wash by hand with neutral soap (max. 30 °C),
- Rinse carefully,
- Allow to dry in the open air,
- Do not dry in a dryer,
- Do not use bleach,
- Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sun light etc. During use or during washing do not use alcohols, ointments or dissolvent liquids. If the orthosis is not properly dried any detergent residues could irritate the skin and deteriorate the product.

Store at room temperature.

In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.

FITTING

POSITIONING POINT:

A Figure 1

- Fit while standing up with your legs slightly apart in line with your hips.
- Move one of your legs away slightly to one side (Fig. 1).
- Locate the hollow between the upper part of the leg and the hip.

This will be the product positioning point.

FITTING:

B Figures 2 and 3

- Place the band in line with the positioning point.
- Check that the pads are centred (Fig. 2).
- Adjust if necessary (Fig. 3).

C Figure 4

- Fasten the band with the velcro (Fig. 4).
- Adjust the tightness if necessary.

D Figure 5

- Then fasten the two posture reinforcement side straps to the desired tightness (Fig. 5).
- Check the alignment and positioning (Fig. 6).

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