- | | |
|---|--|
| <p>D Sprechendes Oberarm-
Blutdruckmessgerät
Gebrauchsanweisung2-12</p> <p>F Tensiomètre parlant pour bras
Mode d'emploi 13-22</p> <p>I Sfigmomanometro parlante da braccio
Istruzioni per l'uso 23-32</p> | <p>NL Spreekende bovenarm-bloeddrukmeter
Gebruiksaanwijzing 33-42</p> <p>GB Speaking upper arm blood pressure
monitor
Instructions for use 43-52</p> <p>Electromagnetic Compatibility
Information 53-54</p> |
|---|--|

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Dear Customer,

thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air. Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,
Your Beurer Team

1. Getting to know your instrument

Check that the packaging of the Beurer BM 49 blood pressure monitor has not been tampered with and make sure that all the required contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

The upper arm blood pressure monitor is used for non-invasive measurement and monitoring of adults' arterial blood pressure.

You can use it to measure your blood pressure quickly and easily, storing the results and displaying the progression of readings together with the average.

A warning is issued for anyone suffering from cardiac arrhythmia.

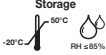
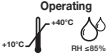
The values determined are classified and graphically evaluated according to WHO guidelines.

Keep these instructions carefully for further use and also let other users have access to them.

2. Important information

Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

	Caution		Manufacturer
	Note Note on important information	Storage 	Permissible storage temperature and humidity
	Follow instructions for use	Operating 	Permissible operating temperature and humidity
	Type BF applied part		Keep dry
	Direct current	SN	Serial number
	Disposal in accordance with EC Directive WEEE (Waste Electrical and Electronic Equipment).	 0483	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products.

Advice on use

- In order to ensure comparable values, always measure your blood pressure at the same time of day.
- Before every measurement, relax for about five minutes.
- If you want to perform several measurements on the same person, wait five minutes between each measurement.
- Do not take a measurement within 30 minutes after eating, drinking, smoking or exercising.
- Repeat the measurement if you are unsure of the measured value.
- The measurements taken by you are for your information only – they are not a substitute for a medical examination! Discuss the measurements with your doctor, and never base any medical decisions on them (e.g. medicines and their administration)!
- Do not use the blood pressure monitor on newborns, pregnant women or patients with preeclampsia. We recommend consulting a doctor before using the blood pressure monitor during pregnancy.
- Cardiovascular diseases may lead to incorrect measurements or have a detrimental effect on measurement accuracy. The same also applies to very low blood pressure, diabetes, circulatory disorders and arrhythmias as well as chills or shaking.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Only use the device on people who have the specified upper arm measurement for the device.
- Please note that when inflating, the functions of the limb in question may be impaired.
- During the blood pressure measurement, blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions, remove the cuff from the arm.
- Avoid any mechanical restriction, compression or bending of the cuff line.
- Do not allow sustained pressure in the cuff or frequent measurements. The resulting restriction of the blood flow may cause injury.
- Ensure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intravascular access or therapy, or an arteriovenous (AV) shunt.
- Do not use the cuff on people who have undergone a mastectomy.
- Do not place the cuff over wounds as this may cause further injury.
- You can either use the blood pressure monitor with batteries or with a mains part. Please note that data transfer and data storage is only possible when your blood pressure monitor is supplied with power. As soon as the batteries are empty or the mains part is disconnected from the power supply, the blood pressure monitor loses the date and time.
- To conserve the batteries, the monitor switches off automatically if no buttons are pressed for one minute.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

Storage and Care

- The blood pressure monitor is made up of precision electronic components. Accuracy of readings and the instrument's service life depend on careful handling.
 - You should protect the device from impact, moisture, dirt, major temperature fluctuations and direct exposure to the sun's rays.
 - Never drop the device.
 - Do not use near strong electromagnetic fields, i.e. keep it away from any radio systems and mobile phones.
 - Only ever use the cuffs provided with the monitor or original replacement cuffs. Otherwise erroneous results will be recorded.
- Do not press any buttons until the cuff is in position.
- If the instrument is not used for any length of time, we recommend removing the batteries.

Advice on batteries


- Batteries can be fatal if swallowed. You should therefore store the batteries and products where they are inaccessible to small children. If a battery has been swallowed, call a doctor immediately.
- Batteries should not be charged or reactivated with any other means, nor should they be taken apart, thrown in the fire or short-circuited.
- Remove the batteries from the instrument if they are worn out or if you are not going to use the instrument for any length of time. This prevents any damage as a result of leakage. Always replace all the batteries at the same time.
- Never use different types of battery, battery brands or batteries with different capacities. You should preferably use alkaline batteries.

Repair and disposal

- Batteries do not belong in domestic refuse. Used batteries should be disposed of at the collection points provided.
- Never open the instrument. If these instructions are not heeded, the warranty will be null and void.
- Never attempt to repair the instrument or adjust it yourself. We can no longer guarantee perfect functioning if you do.
- Repairs may only be performed by Customer Service or authorized dealers. However, always check the batteries and replace them if necessary prior to making any complaint.
- The appliance should be disposed of according to Regulation WEEE (Waste Electrical and Electronic Equipment). In case of queries, please contact the municipal authorities responsible for waste disposal in your area.

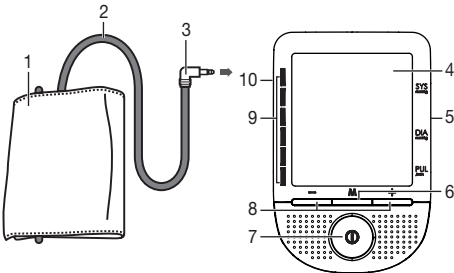


3. Unit description

1. Cuff
2. Cuff line
3. Cuff connector
4. Display
5. Connection for mains part
6. Memory button **M**
7. START/STOP button 
8. Function buttons **-/+**
9. WHO scale
10. Connection for cuff connector

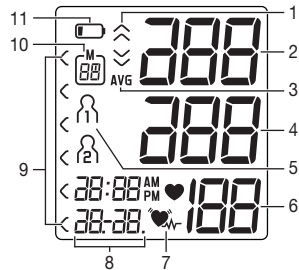Information on the display:

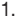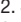
1. Inflate , release air 
2. Systolic pressure
3. Memory display: average value (**AVG**), morning (**AM**), evening (**PM**)
4. Diastolic pressure
5. User memory
6. Calculated pulse value
7. Cardiac arrhythmia symbol 
8. Time and date
9. WHO classification
10. Memory space number
11. Battery replacement symbol 

4. Preparing the measurement

Insert the batteries


- Open the battery compartment lid.
- Insert four 1.5V AA (alkaline type LR6) batteries. Make sure that the batteries are inserted with the correct polarity, according to the label. Do not use rechargeable batteries. The message *"The device is ready for use. You may start measuring."* is displayed.
- Close the battery compartment lid again carefully.
- With inserted batteries, the unit permanently displays the time and date.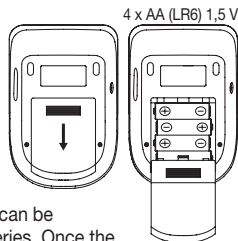

If the low battery indicator  and **EE** appear, no further measurements can be performed. The device will announce: *"Battery weak."* Please replace all batteries. Once the batteries have been removed from the unit, the date, time and language must be set again. Used batteries should not be disposed of in normal household waste. Dispose of them via your electronics retailer or your local recycling point. You are legally obligated to do so.

Note: The codes below are printed on batteries containing harmful substances:

Pb: battery contains lead, Cd: battery contains cadmium, Hg: battery contains mercury.

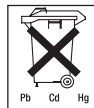Set date, time and language

This menu allows you to set the following functions, one after another.

Date → Time → Language → Volume

It is essential to set the date and time. Otherwise, you will not be able to save your measured values correctly with a date and time and access them again later.

The time is displayed in 24-hour format.

In addition, the device has 3 language settings. The device is set to English when supplied to the customer.

 If you press and hold the function buttons **-/+**, you can set the values more quickly.

- Date/Time**
- Insert the batteries or press and hold the Memory button **M** for 5 seconds.
The year flashes on the display.
 - Set the year with the function buttons **-/+** and confirm with the Memory button **M**.
 - Set the month, day, hour and minutes and confirm each setting with the Memory button **M**.

- Language**
- The language will flash on the display **L 1**.
- You can select the following languages with the function buttons **-/+**:
 - L 1** = German
 - L 2** = French
 - L 3** = Italian
 - L 4** = Dutch
 - L 0** = Language off

Confirm your selection with the Memory button **M**.

 **L 0** = Language off means that there is no voice output and therefore no acoustic playback.

Voice output

*The device is ready for use.
You may start measuring.*

*Deutsch
Français
Italiano
Nederlands*

Volume level **Vo1** flashes on the display.

- You can set the volume of the selected language using the function buttons **-/+**:

Vo3 = loud

Vo2 = medium

Vo1 = quiet

Confirm your selection with the Memory button **M**.

Operation with the mains part

You can also operate this device with a mains part.

When doing so, there must not be any batteries in the battery compartment. The mains part can be obtained from specialist retailers or from the service address using order number 071.60.

- To prevent possible damage to the device, the blood pressure monitor must only be used with the mains part described here.
- Insert the mains part into the connection provided for this purpose on the right-hand side of the blood pressure monitor. The mains part must only be connected to the mains voltage that is specified on the type plate.
- Then insert the mains plug of the mains part into the mains socket.
- After using the blood pressure monitor, unplug the mains part from the mains socket first and then disconnect it from the blood pressure monitor. As soon as you unplug the mains part, the blood pressure monitor loses the date and time setting but the saved measurements are retained.

5. Measuring blood pressure

Please ensure the unit is at room temperature before measuring.





Attach the cuff

Place the cuff on to the bare left upper arm. The circulation of the arm must not be hindered by tight clothing or similar.

The cuff must be placed on the upper arm so that the bottom edge is positioned 2–3 cm above the elbow and over the artery. The line points to the centre of the palm.

Now tighten the free end of the cuff, but make sure that it is not too tight around the arm and close the Velcro fastener. The cuff should be fastened so that two fingers fit under the cuff.

Now insert the cuff line into the connection for the cuff connector.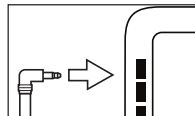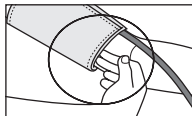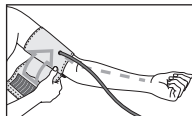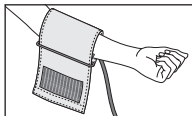


Caution: The unit may only be operated with the original cuff. The cuff is suitable for an arm circumference of 22 to 36 cm.

A larger cuff for upper-arm circumferences of 35 to 44 cm can be obtained from specialist retailers or from the service address using order number 163.387.

Adopt the correct posture

- Rest for approx. 5 minutes before each measurement. Otherwise there may be divergences.
- You can perform the measurement either sitting or lying down. Always make sure that the cuff is on a level with your heart.
- To carry out a blood pressure measurement, make sure you are sitting comfortably with your arms and back leaning on something. Do not cross your legs. Place your feet flat on the ground.
- In order not to distort the result, it is important to keep still during the measurement and not talk.

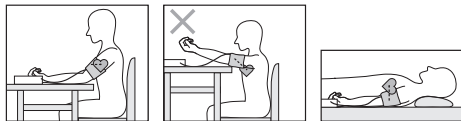Selecting memory

User memory

- Switch on the device with the Start/stop **ⓘ** button.
- **ⓘ** When the device is switched off, the time and date are permanently displayed.
- Select the desired user memory by pressing the function buttons **-/+**.

You have 2 memories, each with 60 memory spaces to store the measurements of 2 different people separately.

Voice output when voice function is active

*The device is ready for use.
You may start measuring.*

*User memory 1
User memory 2*

Performing the blood pressure measurement

Measurement

- As described before, attach the cuff and adopt the posture in which you want to perform the measurement.
- Switch on the device with the Start/stop **ⓘ** button.
- A display check is performed, during which all display segments light up.
- Start the device with the Start/stop **ⓘ** button. The memory space that will be used is displayed.
- The cuff is inflated to 190 mmHg. The cuff's air pressure is slowly released. If a tendency for high blood pressure is already recognised, the cuff is reinflated and the cuff's pressure is increased again. As soon as a pulse is detected, the **♥** symbol flashes.
- **ⓘ** Measurement can be cancelled at any time by pressing the Start/stop button **ⓘ**.
- Systolic pressure, diastolic pressure and pulse measurements are displayed.

Voice output when voice function is active

*The device is ready for use.
You may start measuring.*

Systole ... mmHg

Diastole ... mmHg

Heart rate ... beats per minute

According to the WHO guidelines your blood pressure is

- optimal
- normal
- high to normal
- shows slight hypertension
- shows medium hypertension
- shows strong hypertension

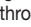
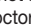
i Whilst the message is being displayed, the volume can be adjusted with the function buttons **-/+**.

- **E** appears if the measurement could not be performed properly. Observe the chapter on error messages/trouble-shooting in these instructions for use and repeat the measurement.
- The measurement is automatically stored.
- The device switches off automatically after 1 minute.

Wait at least 5 minutes before performing another measurement!

6. Evaluating results

Cardiac arrhythmia:

This unit can identify potential disruption of the heart rhythm when measuring and if necessary, indicates this after the measurement with the symbol . If the voice function is activated, the device will announce: "A possible cardiac arrhythmia was found." This can be an indicator for arrhythmia. Arrhythmia is an illness in which the heart rhythm is abnormal because of flaws in the bioelectrical system that regulates the heartbeat. The symptoms (skipped or premature heart beats, pulse being slow or too fast) can be caused by factors such as heart disease, age, physical make-up, excess stimulants, stress or lack of sleep. Arrhythmia can only be determined through an examination by your doctor. If the symbol  is shown on the display after the measurement has been taken, it should be repeated. Please ensure that you rest for 5 minutes beforehand and do not speak or move during the measurement. If the symbol  appears frequently, please consult your doctor. Self-diagnosis and treatment based on the measurements can be dangerous. Always follow your GP's instructions.

WHO classification:

In accordance with the guidelines/definitions of the World Health Organization, the measurements are classified and assessed according to the following table:

Blood pressure value category	Systole (in mmHg)	Diastole (in mmHg)	Action
Grade 3: severe hypertension	≥ 180	≥ 110	seek medical attention
Grade 2: moderate hypertension	160–179	100–109	seek medical attention
Grade 1: mild hypertension	140–159	90–99	regular monitoring by doctor
High normal	130–139	85–89	regular monitoring by doctor
Normal	120–129	80–84	self-monitoring
Optimal	< 120	< 80	self-monitoring

Source: WHO, 1999

The bar chart on the display and the scale on the unit show which category the recorded BMI values fall into.

If the values of systole and diastole fall into two different WHO categories (e.g. systole in the 'High normal' category and diastole in the 'Normal' category), the graphical WHO classification on the unit always shows the higher category; for the example given this would be 'High normal'.

7. Saving, displaying and deleting measurements

i Whilst the message is being displayed, the volume can be adjusted with the function buttons **-/+**.

User memory

The results of every successful measurement are stored together with the date and time. If there are more than 60 measurements, the oldest measurements are lost.

- Switch on the device with the Start/stop **Ⓚ** button.
- Select the desired user memory (M_1 or M_2) by pressing the function buttons **-/+**.
- Press the Memory button **M**. The average value of all saved measured values in this user memory is displayed first **AVG**.
- If you press the Memory button **M** again, the average value of the morning measurements for the last 7 days will be displayed (morning: 5 a.m. – 9 a.m., display **AM**).
- If you press the Memory button **M** again, the average value of the evening measurements for the last 7 days will be displayed (evening: 6 p.m. – 8 p.m., display **PM**).
- If you continue to press the Memory button **M**, the most recent individual measured values are displayed in turn with the date and time.

Average values

Individual measured values

Delete measured values

- The measured values in the corresponding user memory can be deleted by switching on the device, selecting the desired user memory and pressing the function buttons **-** and **+** simultaneously for 5 seconds.
- The device will go directly into standby mode (clock mode).

Voice output when voice function is active

*The device is ready for use.
You may start measuring.*

*User memory 1
User memory 2*

*Average count:
Systole ... mmHg
Diastole ... mmHg
Heart rate ... beats per minute
According to the WHO guidelines your blood pressure is ...*

*Memory space ...
Systole ... mmHg
Diastole ... mmHg
Heart rate ... beats per minute
According to the WHO guidelines your blood pressure is ...*

All the counts in the memory have been erased.

8. Error message/trouble shooting

In the event of errors, the error message E_n appears on the display.

Error messages can occur when

1. it was not possible to record the pulse (E_1),
2. you move or speak during the measurement (E_2),
3. the cuff is fastened too tightly or loosely (E_3),
4. errors occur during the measurement (E_4),
5. the pump pressure is higher than 300 mmHg (E_5),
6. the batteries **🔋** are almost empty (E_6).

In such cases, repeat the measurement. Ensure that you do not move or speak.
If necessary, reinsert or replace the batteries.

9. Cleaning and storing the unit

- Clean your blood pressure monitor carefully using a slightly damp cloth only.
- Do not use any cleaning agents or solvents.
- Under no circumstances should you hold the unit under water, as this can cause liquid to enter and damage the unit.
- If you store the unit, no heavy objects should be placed on top of it. Remove the batteries. The cuff line should not be bent sharply.

10. Technical specifications

Model no.	BM 49
Measurement method	Oscillometric, non-invasive blood pressure measurement on the upper arm
Measurement range	Cuff pressure 0–300 mmHg, systolic 50-250 mmHg, diastolic 30-200 mmHg, Pulse 30–180 beats/minute
Display accuracy	Systolic ± 3 mmHg, diastolic ± 3 mmHg, pulse $\pm 5\%$ of the value shown
Measurement inaccuracy	Max. permissible standard deviation according to clinical testing: systolic 8 mmHg/diastolic 8 mmHg
Memory	2 x 60 memory spaces
Dimensions	L 145 mm x W 96 mm x H 60 mm
Weight	Approx. 264 g (without batteries)
Cuff size	22 to 36 cm
Permissible operating conditions	+10 °C to +40 °C, $\leq 85\%$ relative air humidity (non-condensing)
Permissible storage conditions	-20 °C to +50 °C, $\leq 85\%$ relative air humidity, 800–1050 hPa ambient pressure
Power supply	4 x 1,5V  AA batteries
Battery life	For approx. 250 measurements, depending on the blood pressure level and/or pump pressure
Accessories	Instruction for use, 4 x 1.5V AA batteries, storage pouch
Classification	Internal supply, IPX0, no AP or APG, continuous operation, type BF applied part

Technical information is subject to change without notification to allow for updates.

- This unit is in line with European Standard EN 60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this unit. More details can be requested from the stated Customer Service address or found at the end of the instructions for use.
- This device is in line with the EU Medical Devices Directive 93/42/EC, the „Medizinproduktegesetz“ (German Medical Devices Act) and the standards EN 1060-1 (non-invasive sphygmomanometers, Part 1: General requirements), EN 1060-3 (non-invasive sphygmomanometers, Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and IEC 80601-2-30 (Medical electrical equipment – Part 2–30: Particular requirements for the safety and essential performance of automated non-invasive blood pressure monitors).
- The accuracy of this blood pressure monitor has been carefully checked and developed with regard to a long useful life. If using the device for commercial medical purposes, it must be regularly tested for

accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.

11. Mains part

Model no.	FW 7575M/EU/6/06
Input	100–240V, 50–60 Hz
Output	6V DC, 600 mA, only in connection with beurer blood pressure monitor.
Supplier	Friwo Gerätebau GmbH
Protection	This device is double insulated and protected against short circuit and overload by a primary thermal fuse. Make sure to take the batteries out of the compartment before using the mains part.
	Polarity of the the DC voltage connection
	Double insulated/equipment class 2
Enclosures and Protective Covers	Equipment enclosed to protect against contact with live parts, and with parts which can become live (finger, pin, hook test). The operator shall not contact the patient and the output plug of AC mains part simultaneously.

Electromagnetic Compatibility Information

Guidance and manufacture's declaration – electromagnetic emissions – for all EQUIPMENT and SYSTEMS

Guidance and manufacture's declaration – electromagnetic emission

The BM49 blood pressure monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the BM49 blood pressure monitor should assure that it is used in such an environment.

Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The BM49 blood pressure monitor uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	The BM49 blood pressure monitor is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.

Guidance and manufacture's declaration – electromagnetic immunity – for all EQUIPMENT and SYSTEMS

Guidance and manufacture's declaration – electromagnetic immunity

The BM49 blood pressure monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the BM49 blood pressure monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floor are covered with synthetic material, the relative humidity should be at least 30%.
Power frequency (50Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Guidance and manufacture's declaration – electromagnetic immunity – for EQUIPMENT and SYSTEMS that are not LIFE-SUPPORTING

Guidance and manufacture's declaration – electromagnetic immunity

The BM49 blood pressure monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the BM49 blood pressure monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 V _{rms} 150 kHz to 80 MHz	3 V _{rms}	<p>Portable and mobile RF communications equipment should be used no closer to any part of the BM49 blood pressure monitor, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance:</p> $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$ $d = \left[\frac{3.5}{E_1} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[\frac{7}{E_1} \right] \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	<p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>

Guidance and manufacture's declaration – electromagnetic immunity

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the BM49 blood pressure monitor is used exceeds the applicable RF compliance level above, the BM49 blood pressure monitor should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the BM49 blood pressure monitor.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the EQUIPMENT or SYSTEM – for EQUIPMENT or SYSTEM that are not LIFE-SUPPORTING

Recommended separation distances between portable and mobile RF communications equipment and the BM49 blood pressure monitor

The BM49 blood pressure monitor is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the BM49 blood pressure monitor can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BM49 blood pressure monitor as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2.5 GHz
	$d = \left[\frac{3.5}{V_r} \right] \sqrt{P}$	$d = \left[\frac{3.5}{E_r} \right] \sqrt{P}$	$d = \left[\frac{7}{E_r} \right] \sqrt{P}$
0.01	0.117	0.117	0.234
0.1	0.370	0.370	0.740
1	1.170	1.170	2.340
10	3.700	3.700	7.400
100	11.7	11.7	23.4

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.