geemarc™ CL7400 OPTI

Hear the Difference

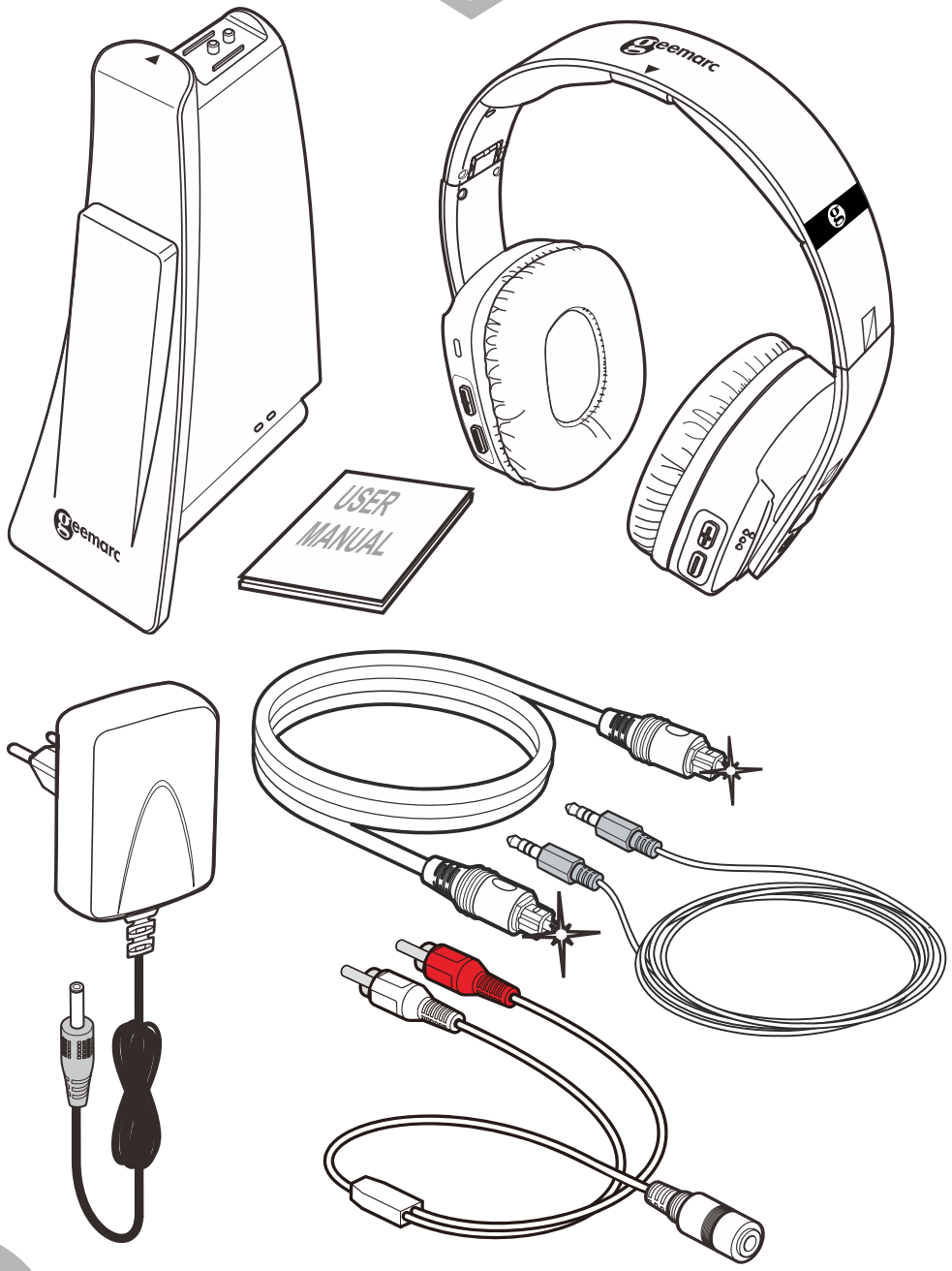
En	2.4 GHz Wireless Digital Quality Sound
Fr	Casque sans fil 2.4 GHz Son Qualité HiFi
D	2.4 GHz Funk-Kopfhörer mit kristallklarem Stereosound
Español	Control remoto inalámbrico de 2,4 GHz con sonido de calidad digital
Italiano	Cuffia a 2.4 GHz wireless qualità del suono HiFi
Nederlands	2,4 GHz draadloos en digitaal kwaliteitsgeluid
עברית	2.4GHz אוזניות שמע דיגיטליות אלוווטיות עם צליל באיכות גבוהה
Svenska	2,4 GHz trådlös digital ljudkvalitet
Magyar	2.4 GHz-es vezeték nélküli technológia, minőségi hangzás
Dansk	2,4 GHz Trådløs digital kvalitetslyd
Norsk	Trådløse hodetelefoner 2,4 GHz med Hi-Fi-lydkvalitet
PolSKI	Bezprzewodowa cyfrowa transmisja dźwięku(2.4 GHz)

Optical input
Entrée optique
Optischer Eingang
Optische invoer
Entrada óptica
Input ottico

En	Unpacking	It	Estrarre dall'imballaggio	Magyar	Kicsomagolás
Fr	Contenu du Colis	NL	Uitpakken	Dansk	Udpakning
De	Auspacken	עברית	פריקה	Norsk	Innholdet i pakken
Esp	Desempaquetar	Svenska	Packa upp	Polski	Zawartość

a

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

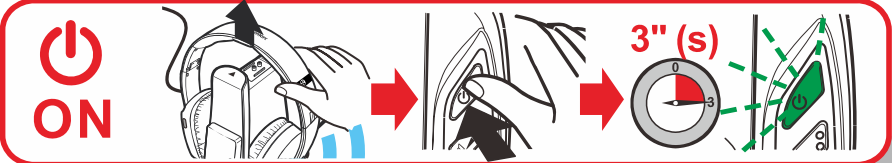
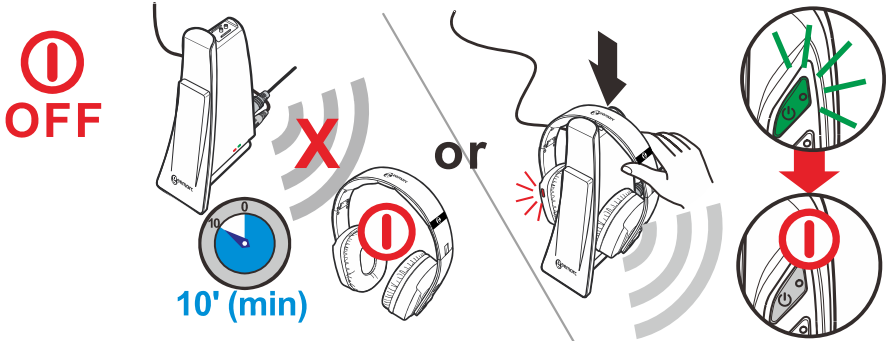
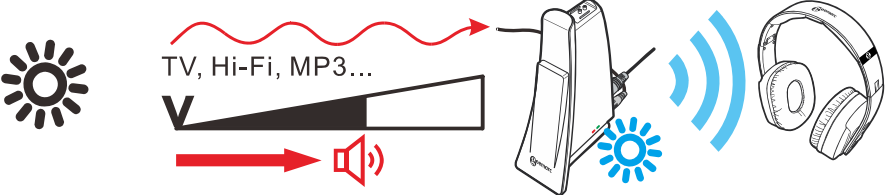
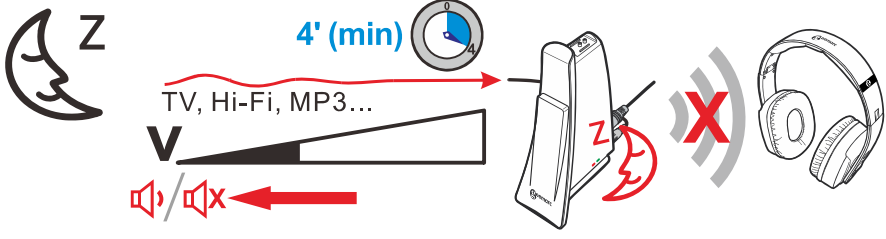
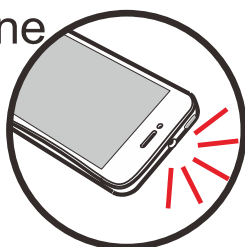
MP3

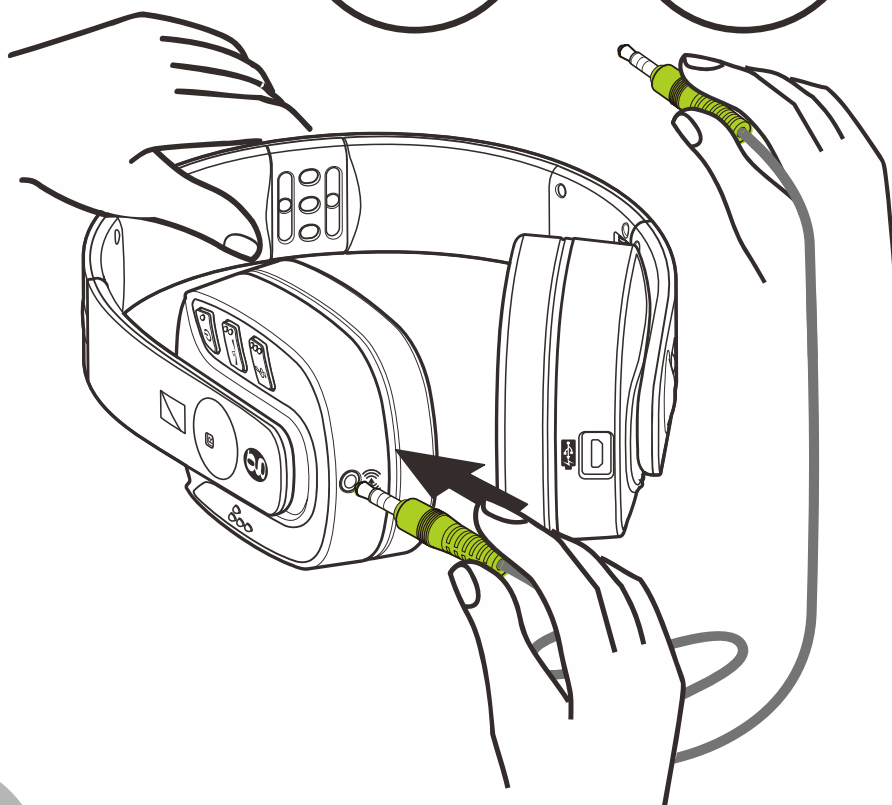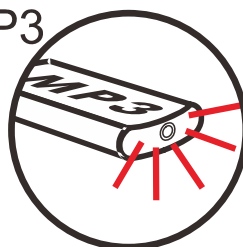Safety & Care advice

To make the best use of this product and to use it safely, please read these instructions thoroughly before use and keep safe for future reference.

Warning: This product can be very loud. Take care to prevent others from using this product, if they do not have the same level of hearing loss. Excessive sound pressure from earphones and headphones can cause hearing loss.

Caution: Electric shock. DO NOT insert unattached plug heads in to mains power sockets. Plug heads **MUST** be fitted to the power adaptor.

Electrical connection : The unit is designed to operate from a 100-240VAC - 50/60Hz supply. (Classified as 'hazardous voltage' according to EN62368-1 standard).

The unit does not incorporate an integral power on/off switch. To disconnect the power, either switch off supply at the mains power socket or unplug the AC adaptor. When installing the unit, ensure that the mains power socket is readily accessible.

Caution: Risk of explosion if battery is replaced by an incorrect type. Dispose of used batteries according to the instructions.

DO NOT cover the ventilation.

DO NOT remove the ear pad as it is very difficult to replace.

Keep the unit away from heat sources.

DO NOT expose the unit to direct sunlight.

Clean the Headset and ear pads regularly.

DO NOT modify the unit in any way.

Please note: The unit may be subject to interference from appliances that generate strong electrical or magnetic fields e.g. microwaves, mobile phones, lighting transformers etc.

Battery Warnings:

- **DO NOT** attempt to dismantle.
- **DO NOT** dispose of in fire.
- **DO NOT** swallow.
- Keep away from children.
- Batteries are to be inserted with the correct polarity.
- **DO NOT** short-circuit the supply terminals.
- Exhausted batteries are to be removed from the product.
- **Do NOT** use another type of battery or non-rechargeable batteries to replace the rechargeable batteries.
- When the headphones are not in use or the batteries are charging, switch **OFF** the headphones to avoid damaging the batteries.
- The batteries must be fully charged when use for the first time. An initial 5 hours charge is required.

Troubleshooting:

NO SOUND

1. Ensure that the mains adapter is pushed fully into the power outlet and that its cable is properly connected to the DC IN 6V connector on the transmitter.
2. Ensure that the headphones On/Off switch is switched to the On Position and the switch is illuminant Green.
3. The headphones battery charge level may be too low. please replace the headset back on it's cradle to recharge the batteries until the green light is illuminated on the left hand headset.
4. Ensure that your DVD, TV set, Hi-Fi system or audio component is switched on and set to a channel that broadcasts an audio signal.
5. Use the headphones volume control to increase the sound level (only when use as a wireless headphone).
6. The audio signal input level is too low. Increase the volume from the audio source of Hi-Fi, CD, or TV, etc.
7. The connected audio/video equipment may not be in playback mode. Start playback on by the equipment.
8. The headphones output level is set too low. Adjust the volume to a suitable level.

DISTORTION

9. Restart the Pairing process. Press the "Pair" button 3sec on the transmitter and the headphone. The process is finished when both LEDs are in Green.
10. The headphone's batteries charge level may be too low. Recharge the batteries.
11. Ensure that the volume setting is correct.
12. The headphones may be too far away from the transmitter. Move closer to it.
13. The audio signal input level is too strong. decrease the volume from the audio source of Hi-Fi, CD, or TV, etc.

Customer support

For product support and help, visit our website at www.geemarc.com

E-mail: help@geemarc.com

For our Customer Helpline: Telephone: **01707 387602**

(lines are open 09h00 to 16h00 Mon to Fri)

General information

Features

- 2.4GHz FHSS digital technology with equalizer features for tone and balance adjustments.
- High quality stereo sound that is virtually interference-free
- Transmission is not restricted solely to line of sight
- Digital Pairing System on Headphone & Transmitter
- Electronic volume control on headphones
- Built-in Pin-charger on transmitter
- Auto shut-off on headphone
- Auto On/Off RF function on transmitter
- 3.5mm Line in feature for wired headphone application
- Operating distance up to 22 meters

Technical specifications

Transmission mode	: 2.4GHz digital FHSS stereo
Carrier frequency	: 2.4GHz
Operating voltage	: Transmitter 6 V  0.6 A : Receiver 3.7V 1200mA (built-in Lithium rechargeable battery)
Frequency response	: 20 Hz – 20 KHz (electrical)
Distortion	: 0.5 %
Signal-to-noise ratio	: 75 dB
Channel separation	: 60 dB
Range	: 22 meters (open area)
Maximum RF Power	: 10dBm
Frequency band	: 2.406GHz – 2.472GHz

Guarantee

From the moment your Geemarc CL7400 OPTI is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our Helpline or visit our website at www.geemarc.com.

The guarantee does not cover accidents, negligence or breakage to any parts.

The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

The product is covered by the legal guarantee of conformity as provided by applicable law.

Important: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only.

EEC Declaration

Geemarc Telecom SA hereby declares that this CL7400 OPTI is in compliance with the essential requirements and other relevant provisions of the Radio Equipment Directive 2014/53/UE.

The declaration of conformity may be consulted at www.geemarc.com

To prevent possible hearing damage, DO NOT listen at high volume levels for long periods.

Disposal information

Disposal of old electrical & electronic equipment (applicable in the European Union and other European countries with separate collection systems)

The symbol on the product or its packaging indicates that this product may not be treated as household waste. Instead it should be handed over to the applicable collection point for the recycling of electrical and electronic equipment.

By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product.

For more detailed information about recycling this product, please contact your local council office, your household waste disposal service or the shop where the product was purchased.

All electrical and electronic products including batteries should be disposed of separately from the municipal waste stream via designated collection facilities appointed by the government or the local authorities.

2 rue Galilée, Parc de l'Etoile, 59791 Grande-Synthe Cedex, France

This product is made in China To Geemarc Telecom

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