VERMEIREN

Vera

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: B, 2015-11

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© N.V. Vermeiren N.V. 2015

EN Contents

1. Technical details 2
2. Components 2
3. Explanation of symbols 2
4. Check after delivery 2
5. Intended use 2
6. Assembly / Disassembly transfer board 3
7. Use of the transfer board 3
 - 7.1. Mounting the transfer board to the bathtub 3
 - 7.2. Removing the transfer board from the bathtub 3
 - 7.3. Turning or removing transfer board 3
 - 7.4. Transfer to/from transfer board 4
8. Carrying the transfer board 4
9. Safety instructions 4
10. Maintenance, inspection, cleaning 4

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INSTRUCTION MANUAL
Rotating tub transfer board
"Vera"

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

| | |
|--|--|
| Make | Vermeiren |
| Address | Vermeirenplein 1/15, B-2920 Kalmthout |
| Type | Rotating tub transfer board |
| Model | Vera |
| Width | 350 mm |
| Length | 730 mm |
| Height | 100 mm |
| Dismantled height | 40 mm |
| Diameter turning disc | 325 mm |
| Distance swivel disc to handgrip | 200 - 300 mm |
| Inner width bathtub (adjustment attachment tips) | 560 - 680 mm |
| Weight total | 3,75 kg |
| Patient weight | max. 130 kg |
| Storage temperature | +5°C - +41°C |
| We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg | |

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the transfer board, read the instruction manual carefully: it will familiarise you with this product.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can contact the dealer for the instructions for use.

2. Components

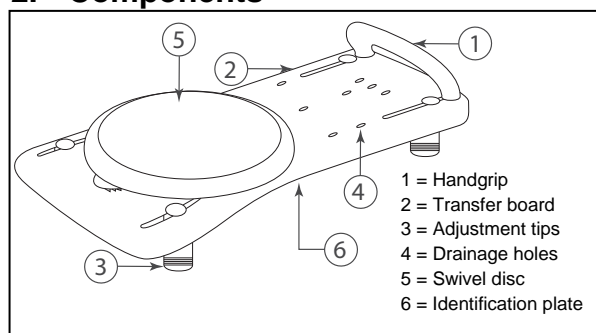
Fig 1. Components

3. Explanation of symbols

Maximum mass


CE conformity

Type designation

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- Transfer board + swivel disc **(A)**
- 4 Adjustment tips **(B)**
- 4 Screws **(C)**
- 1 Handgrip **(D)**
- 2 screws **(E)**
- Manual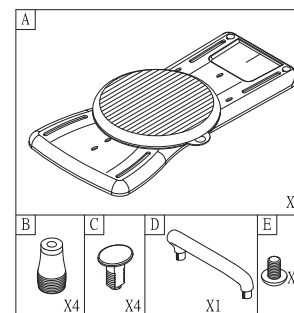

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

5. Intended use

The transfer board shall be used by 1 person. The transfer board shall be used indoor, in a bathtub.

Is your bathtub too small for a conventional tub / shower safety / transfer seat? Then the swivel bath transfer board is the solution. The transfer board is ideal for people who feel insecure to sit on a bath board. The transfer board can be used for a bathtub with inner width of 560 until 680 mm.

The rotating tub transfer board is a 360° turning bath disc that allows easy transfer over the bath board. The board can be locked into four 90 degree positions. The user sits on the swivel disc from the side of the bath and then swivels over the bath for a wash or shower.

The construction of the board allows full use by elderly people or during revalidation in home environment. Not to be used for persons that has no walking ability.

The board shall not be used as toy for children.

The board should only be used on horizontal, non smooth surfaces of the bathtub board. The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

6. Assembly / Disassembly transfer board

⚠ CAUTION: Risk of injury - Check that the transfer board is well assembled.

Mount / remove all the components as showed in the figure below.

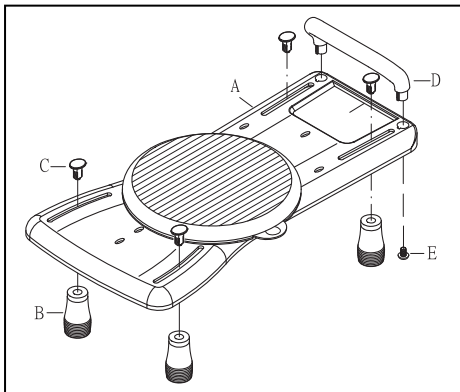
Fig 2. Assembly / Disassembly transfer board

7. Use of the transfer board

The transfer board is ideal for usage in a bathtub.

The transfer board can be mounted on the top of the bathtub and can be easily removed when not in use. So other persons can use the bath.

7.1. Mounting the transfer board to the bathtub:

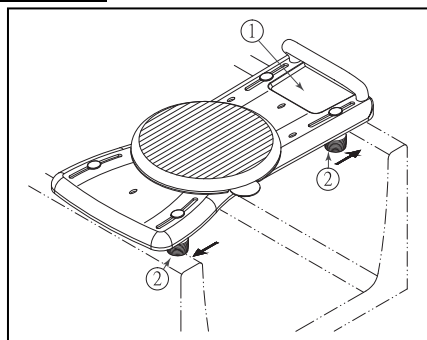
Fig 3. Mounting / removing transfer board to the bath

- ⚠ CAUTION: Risk of injury - Check that the board is properly fixed in the bathtub with the attachment tips ②.**
- ⚠ CAUTION: Risk of injury - Check that the board can rest enough on the bathtub, that the inner width of the bathtub is not too large.**
- ⚠ CAUTION: Risk of injury - Be careful you do not place your fingers, clothes between the bathtub and transfer board when mounting the board in the bathtub.**

1. Place the transfer board ① on the bathtub.

2. Move the attachment tips ② and position these against the bathtub sides.
3. Check that the transfer board is properly fixed to the bathtub.

7.2. Removing the transfer board from the bathtub:

1. Loosen the attachment tips ② from the bathtub.
2. Take the transfer board out the bathtub.

7.3. Turning or moving transfer board:


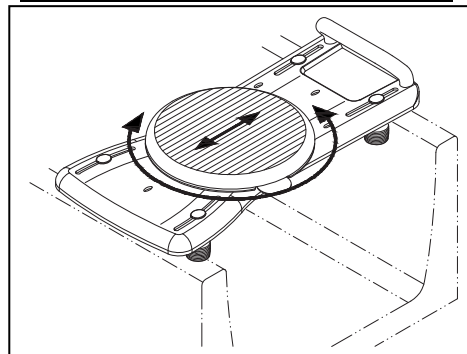
Fig 4. Turning / moving the transfer board

⚠ CAUTION: Risk of injury - Check that the swivel disc is properly locked after turning.

Let someone assist you for a smoothly, safely turning of the transfer board.

1. Pull the plastic lip out the hole.
2. Lift the legs of the patient up when turning to the washing position.
3. Turn the swivel disc to the desired position. The swivel disc can be locked in four 90° positions by placing the lip in one of the four locking holes. Move the swivel disc in a straight direction more to the handgrip or away from the handgrip (range 100 mm: stepless).
4. Lock the swivel disc by placing the swivel disc in the correct locking hole.
5. Check that the swivel disc is properly locked.

Fig 4. Turning / Moving the transfer board

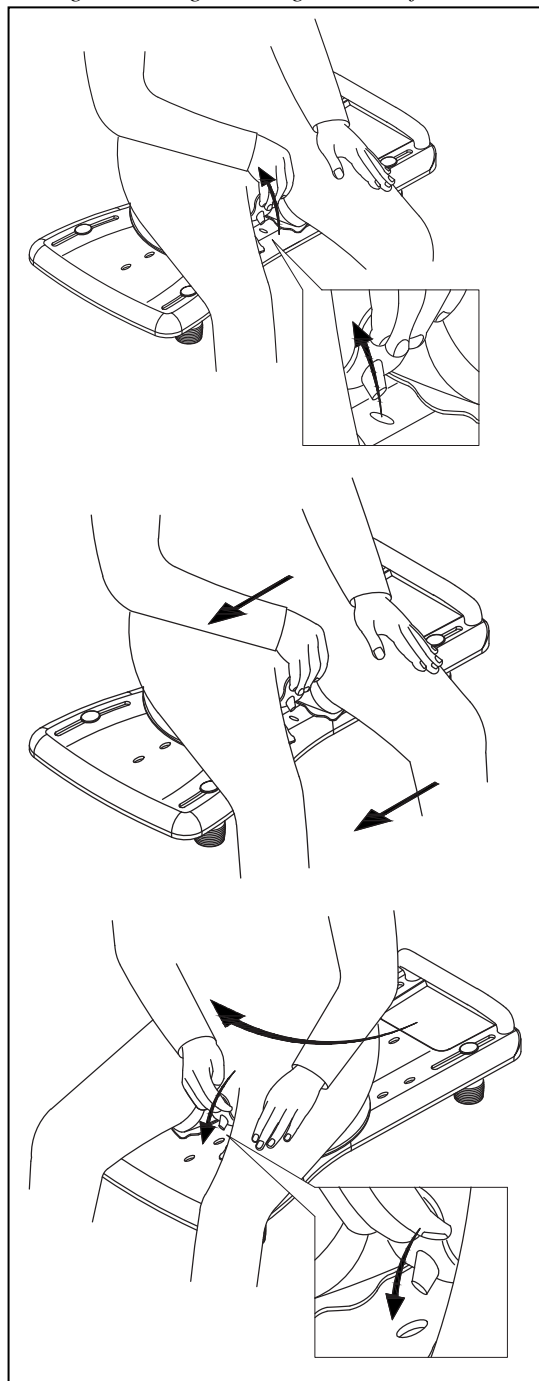**7.4. Transfer to/from transfer board:**

⚠ CAUTION: Risk of injury - Be careful that you not slip out on the bathroom floor.

1. Walk to the transfer board.
2. Place yourself on the board. For extra support you can grasp the handgrip. **Be careful that you not slip out on the floor.**
3. Turn yourself in the bad for the washing position.

If you feel uncertain, you can ask someone to assist you.

8. Carrying the transfer board

The best way to carry the transfer board is to grasp the handgrip.

9. Safety instructions

- **Do not exceed the maximum user weight. The maximum user weight for the transfer board is 130 kg.**
- Use the transfer board on horizontal, non-smooth surfaces of the bathtub board.
- Keep it away from children. The transfer board shall not be used as a toy.
- When it is used by, on or near children, invalids or disabled persons close supervision is necessary.
- Be careful you do not place your fingers, clothes between the bathtub and transfer board when mounting the board in the bad.
- Check that the board can rest enough on the bad board, that the inner width of the bathtub is not too large.
- Check that the board is properly fixed in the bathtub with the attachment tips.
- Do not use the transfer board when it is not working properly.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

10. Maintenance, inspection, cleaning

For the maintenance manual of the transfer board refer to the Vermeiren website: www.vermeiren.com.

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CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multipoition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

- This warranty is not valid in case of:
- damage due to incorrect usage of the wheelchair,
 - damage during transport,
 - involvement in an accident,
 - a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
 - normal wear of the wheelchair,
 - non-return of the warranty card

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Vermeiren Austria GmbH
Vermeiren Suisse S.A.